Eng.: Instruction for Use / Operation Manual

Fr.: Manuel

It.: Manuale

Esp.: Manual

CE
0123

Contents

Definitions and general information.....	4
Purpose of the physiokey	4
Indications	4
Areas of application	4
Contraindications and areas in which use is prohibited	4
Side-effects	5
Safety instructions	5
Commissioning, instruction, safety inspection (STK) and medical devices book, service life	5
Product description	6
Optional components (changeable heads and adhesive pads)	6
Handling.....	8
physiokey Brief Description	8
Explanation of the function of the Home button.....	8
Description Touchscreen Controls	9
Inactive Display.....	9
Locking Times of the Display	10
physiokey Therapy Modes.....	10
Key-phoresis mode	11
Bioregulation mode.....	13
Changing the saved therapy program	15
Individual+ therapy program	16
Diagnostic mode	17
Device Settings	21
Charging the physiokey	22
Replacing the Changeable Electrode Head	23
Care, cleaning and storage	25
Error Messages	26
Recalibration.....	27
Service, repairs and safety inspections (STK)	28
Loss of guarantee and warranty	28
Return and disposal.....	29
Attachment: Contact details and technical information	29
Contact details.....	29
Explanations of the electrical current output of the different therapy modes	29
Key-Phoresis mode	29
Bioregulation mode:	30
Technical data	30
Appliance (physiokey handpiece)	30
Charger (original accessory) (Type: HNP06-USB-L6)	31
Symbols and pictograms.....	31
Results of EMC tests carried out.....	33

Definitions and general information

Purpose of the physiokey

physiokey is an adaptively regulated electrotherapy device.

The device uses (in the “bioregulation” and “diagnostic” modes) the process of non-invasive electrical nerve stimulation which has been established as a successful therapy for many years, through the electrode’s contact with the skin. This process is generally regarded as having very few side-effects.

The physiokey can create a bidirectional (in “bioregulation”) and galvanic impulse form (in “key-phoresis”). The application of galvanic currents with physiokey is used exclusively for the application of topical applications under the skin (see chapter “Key-phoresis mode”).

The application of the bipolar impulse form is discussed below:

The bipolar impulses in bioregulation serve the following indications and areas of application:

Indications

Indications for the physiokey are acute and chronic pain, muscular and other organic dysfunctions (in terms of restrictions in functionality) including muscular imbalances (in terms of restrictions in the functionality of muscle pairs) as well as disorders or restrictions to the blood circulation in relation to microcirculation in the tissue for the areas of application:

Areas of application

- Nervous system
- Musculoskeletal system
- Digestive tract
- Urogenital area

The product can be used on patients of all age groups. Use on children up to the age of 5, however, is reserved exclusively to experts who have been given the appropriate training. The recommended treatment time using the device is a few minutes up to a maximum of one hour.

The physiokey is a medical product. It can be used by medical professionals as well as at home by laypersons who have been trained in its use by a medical professional.

Contraindications and areas in which use is prohibited

physiokey may not be used in the following cases:

- Use prohibited on injured skin (e.g. open wounds and/or wounds which have not yet completely healed)
- Use prohibited in bodily orifices
- Use prohibited on carriers of active implants (e.g. cardiac pacemakers)
- Use prohibited in the area of the uterus with pregnant women
- Use prohibited on patients with cardiac arrhythmia
- Use prohibited on patients with malignant diseases without consultation with a doctor
- No use on patients who suffer from epilepsy

Side-effects

The **physiokey** is associated with very few side-effects. The following cases can be regarded as side-effects, however:

Symptom intensification: Symptoms may intensify during the treatment period, as the body is “dynamised”. These may include stronger pain, muscle aches and other reactions in isolated cases. In most cases, they fade away after a short time. Should they last longer, please interrupt the treatment for a few days and, if necessary, consult a doctor.

The stainless steels of the electrodes in the exchangeable heads contain small traces of nickel. Patients with a pronounced nickel allergy may therefore experience skin irritation. In this case, healthcare professionals must decide on whether the therapy should be continued, or alternative treatment options should be used.

Safety instructions

- Read the instructions on use (this document) before you use the device for the first time.
- Only use the **physiokey** for its intended purpose.
- The **physiokey** must be kept in such a way that it is inaccessible to children and may not be used by children.
- Do not submerge the **physiokey** in water or hold it under flowing water.
- The device must be checked for integrity before use (handpiece, tablet, base station and power pack (incl. cable)) and otherwise sent in for repair.
- With regard to electromagnetic compatibility (EMC), the device also fulfils the stricter requirements regarding “home environment”, which also apply to many doctors’ and therapists’ practices.
- Not suitable for use in intensive care units
- Portable HF communication devices (radios) of all types must not be operated at a distance of less than 30 cm from the device. An exception is charging a mobile phone on the base station.
- In exceptionally rare cases of high EMC interference, however the system may switch itself off. In such a case, restart the device to continue the therapy.
- Only the power packs that are provided by the manufacturer may be used.
- The device must not be charged if it contains moisture. If external liquids or other substances have been used, the device must be thoroughly cleaned afterwards, especially the charging socket.

Commissioning, instruction, safety inspection (STK) and medical devices book, service life

Ensure that your physiokey is fully charged before using it for the first time. Otherwise, the displays on the charging status may not be correct later on.

The use of the device requires instruction. You can receive instruction on how to use the **physiokey** either from your medical device consultant or through the **KeyAkademie** seminars (subject to payment) or an individually booked in-house training course (subject to payment). A medical device book (MPB) is available from Keytec. The device is subject to a safety inspection (STK). The safety inspection (STK) can only be carried out by professional staff who have been trained in the use of the device. A protocol template is printed on the second page of the MPB. When used at home, the treating therapist is responsible for the instruction and safety inspection (STK) obligations and is required to have been instructed by Keytec. The service life of the **physiokey** is 10 years.

Product description

The **physiokey** consists of the following components:

- Handpiece (appliance) including standard changeable head
- Instructions for use
- Base station
- Charging device with mini-USB connector

Optional components (changeable heads and adhesive pads)

The **physiokey** can be supplemented with optional components (changeable heads and adhesive pads) as regards the specific focal points for its application. The changeable heads consist of the connection to the appliance and the treatment electrodes with their different configurations. The electrodes of the adhesive pads are integrated into the textile and adhesive gel. All treatments according to the intended purpose can be carried out with the standard equipment. Improved access to the body and certain regions can be achieved with the optional components, however.

Highpro Scanner changeable head: Ideal for cross-dermatome treatment and therapy of meridians.

Trigger changeable head: Treatment of the smallest structures, also muscle cords, tendon courses, trigger points.

Matrix changeable head: Specialised for fascia therapy and treatment of connective tissue.


Comb changeable head: For treatment of hairy areas.

Connector cable: Adhesive pad connection.

Adhesive pads: For sticking to the skin.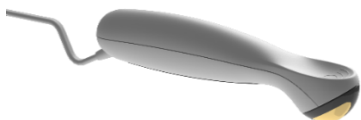

Fascia Flow changeable head: For the treatment of fascia and trigger points.


Hand Comb changeable head: For treatment of hairy areas.

Tabletop base station: Secure storage with charging function.

Handling

physiokey Brief Description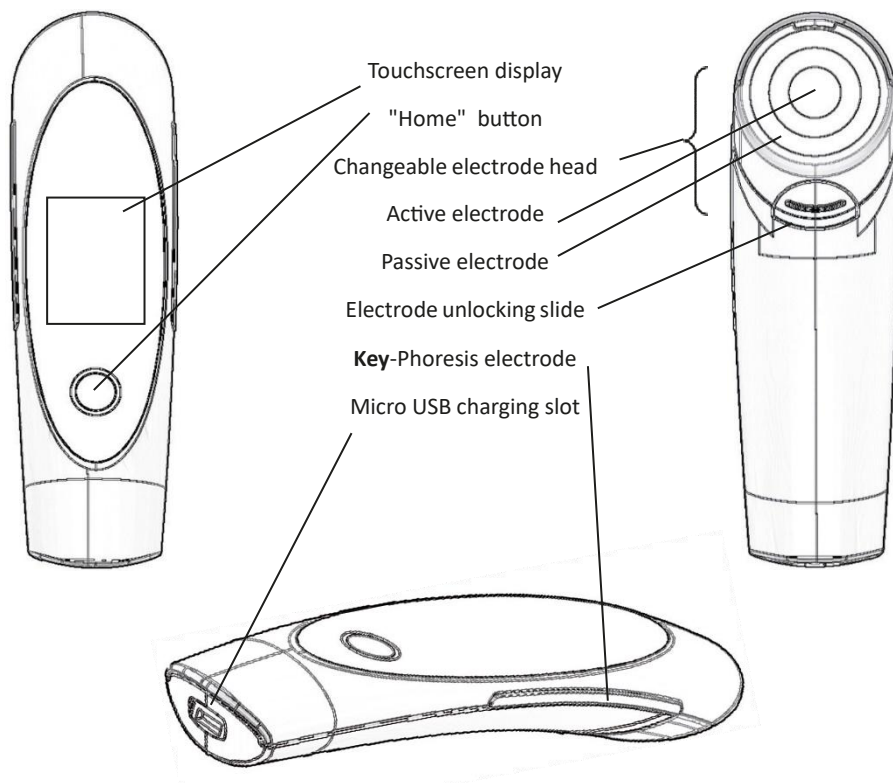

Explanation of the function of the Home button

The Home button is the only "physical" pushbutton on the physiokey, and has 4 functions:


1. **Switch on and off:** With the Home button you switch the device on. To do so, press the button for two seconds. By pressing the button again for two seconds, the device is switched off again. The logo is shown on the screen when both switching the device on and switching it off. Please wait at least 15 seconds before switching the device on again.
2. **Activating the display:** If the touchscreen is inactive, you can activate it by pressing the Home button briefly once.
3. **Return to the main menu:** By pressing once briefly, if the touchscreen is active, you will exit the therapy mode and return to the main menu.
4. **Reset/recalibration:** Is carried out with a blue screen, see "Recalibration" section.

Description Touchscreen Controls

The display of the **physiokey** features touchscreen controls. If the energy setting or the therapy mode is to be changed, simply press the respective symbol or button on the display. In this description, the phrase "press" is often used regarding symbols/buttons. However, light touching is often sufficient. In general, touching the button directly triggers the intended function and is confirmed by an acoustic signal. In a list, however, functions are selected first with the arrow symbols and then started by another selection symbol (see description: Device Settings).

PLEASE NOTE: Handling the touchscreen is a matter of practice. After a few applications, the functions of the touchscreen will be familiar.

PLEASE NOTE: The touch sensitivity can be set in the service menu (see chapter "Device Settings").

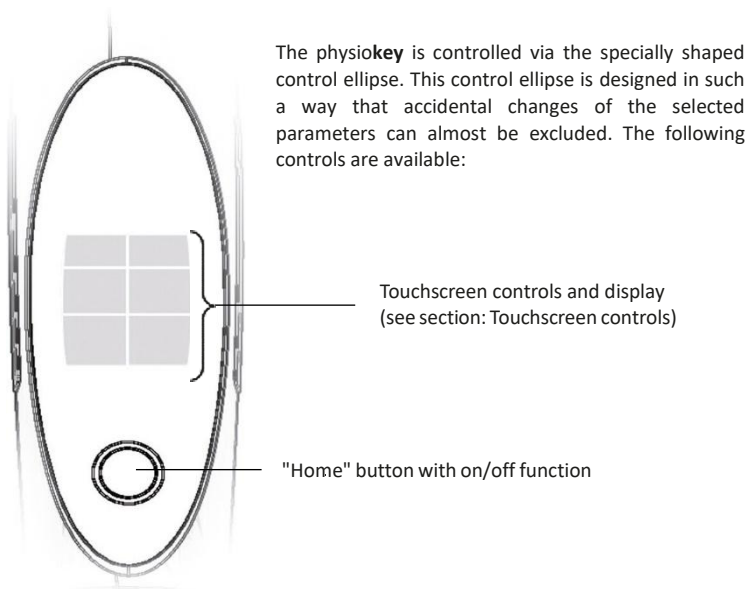Inactive Display

To prevent accidental activation of the touchscreen, the touchscreen functions are subsequently locked if they are not used for a specific period of time. If one section of the touchscreen controls is inactive, the following symbol is displayed at the bottom of the screen:

Additionally, the locked buttons turn white.


Briefly press the "Home" button once to reactivate the touchscreen display. If the display is activated, press the "Home" button once to return to the main menu.

Locking Times of the Display

Already after 4 seconds, only the plus and minus buttons remain active to adjust the energy setting. If the energy setting is not changed for 9 seconds, all touchscreen controls are locked and the display brightness is reduced. Press the "Home" button to fully reactivate the touchscreen controls.

physiokey Therapy Modes

Press the Home button for approx. two seconds to switch on the physiokey. The device will now start up. The main menu appears.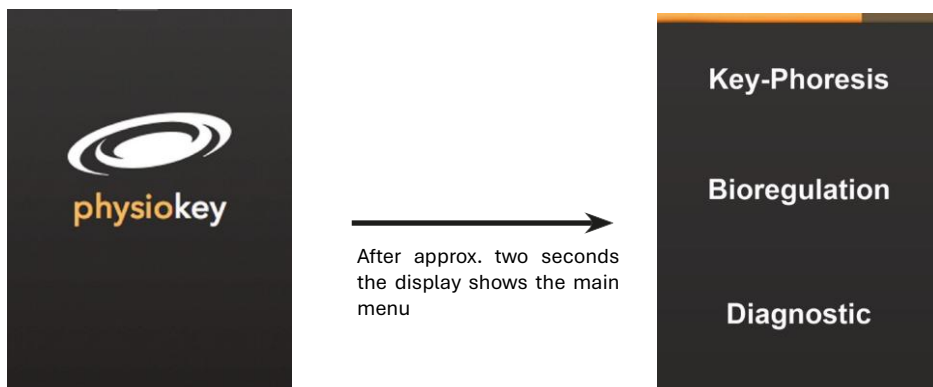

Three therapy modes are available to you in the main menu.

Key-phoresis mode

Select the **key**-phoresis mode by touching the corresponding item in the main menu.

Key-phoresis is a special form of application using the characteristics of direct current. This enables the application of the galvanic procedure, during which a low direct current is produced.


In the case of phoresis, the physio**key** is used as an aid in the case of topical applications. Please note their instructions for use as well as indications and contraindications of the topical application. Please take note of them for both the intended purpose and instructions for use. Also note the indications and contraindications of the topical application.

The changeable head is placed on the skin, the hand electrode is required as a counter-electrode. You touch this automatically when using the device, your contact allows the effect of the electrode to start. To close the circle, there must be skin contact, and ideally, the skin (including the hand) should be slightly moisturised.


**You will not generally feel the key-phoresis.
The required effect is still achieved, though.**

It is also possible for the patient to operate the device themselves for the duration of the **key**-phoresis.


You can activate the **key**-phoresis by pressing the item.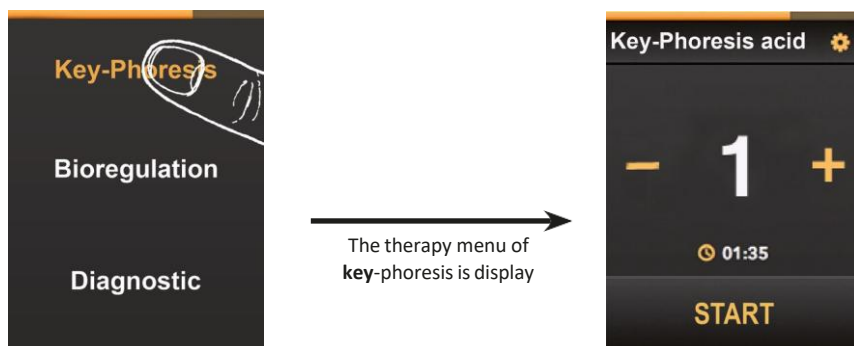

Key-Phoresis mode overview

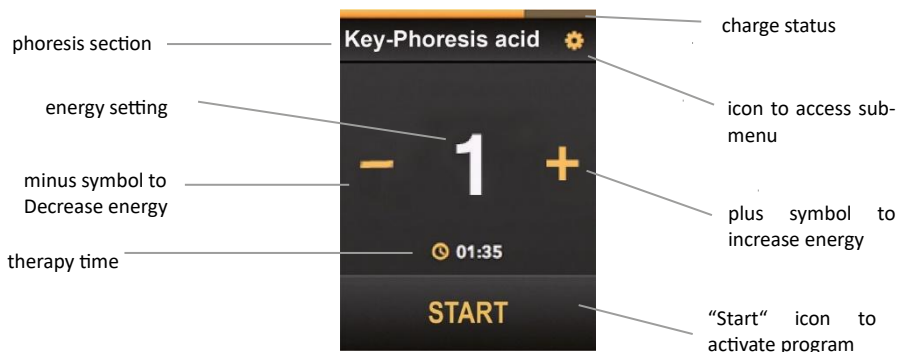To perform **key-phoresis**, increase the energy level first by pressing the plus button. Touch the minus button to reduce the energy. The energy level is indicated by the number in the centre. You can choose from 1-99 for the energy setting. The energy should be barely noticeable or not noticeable at all. After selecting the energy level, press “Start”. The timer starts to run when the power is transmitted. If you want to stop the application, press “Finish”.

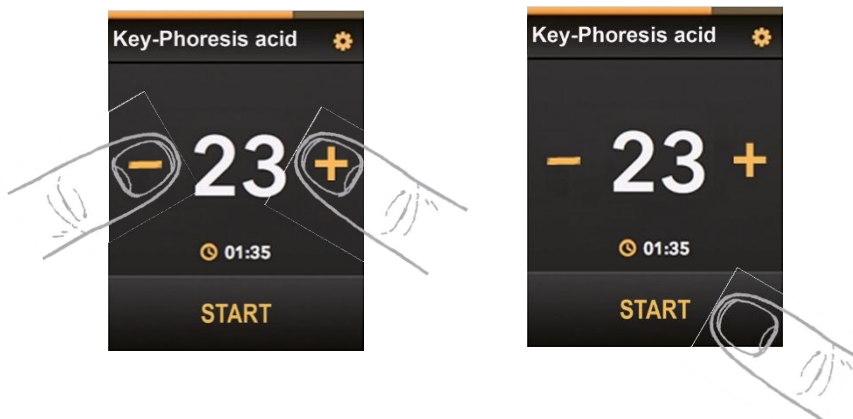
To change the setting from basic to acid, press the gear symbol. Another sub-menu is displayed. Now, the two options are available and can be set by pressing the respective button. The last active setting is indicated by a brighter button.

Bioregulation mode

Select the Bioregulation mode by touching the corresponding item in the main menu.

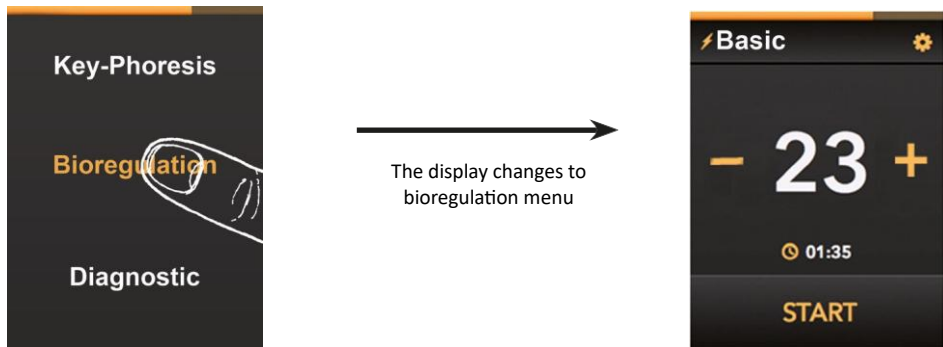
Bioregulation mode overview


If the lightning symbol is displayed in front of a therapy program, the output energy is applied at the electrodes.

Press the plus or minus button to increase or decrease the energy setting. Pressing the buttons is confirmed by an acoustic signal. The energy level is indicated by the number in the centre. You can choose from 1-99 for the energy setting. In most cases, however, it should be set so that it feels comfortable. The basic treatment can also be carried out in this setting. After switching on the physiokey, the therapy program in the Bioregulation mode is always set to "Basic", with the energy level of 1.

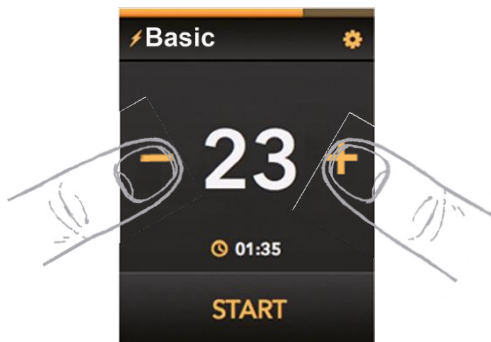
Press "START" to activate a previously saved therapy program. This is indicated by the description above (e.g. Triggering). Press "STOP" to return to basic settings.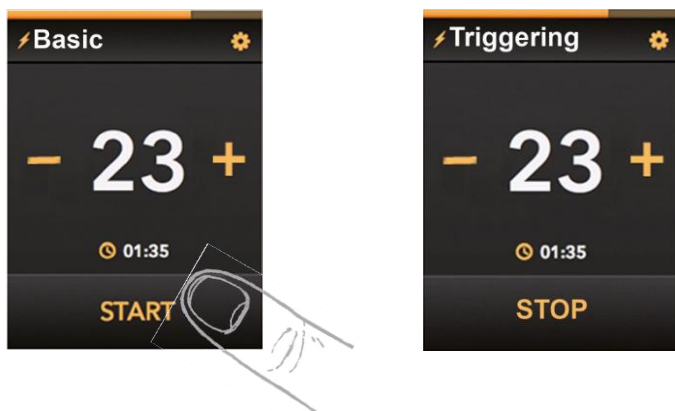

Changing the saved therapy program



With your **physiokey**, a wide range of different therapy programs are available to you. This way, the device offers maximum flexibility. The different therapy programs are named according to the effect to be achieved. For the programs you wish to select in specific cases, please refer to the seminar documentation. The selection is made by pressing the gear symbol to open an additional sub-menu.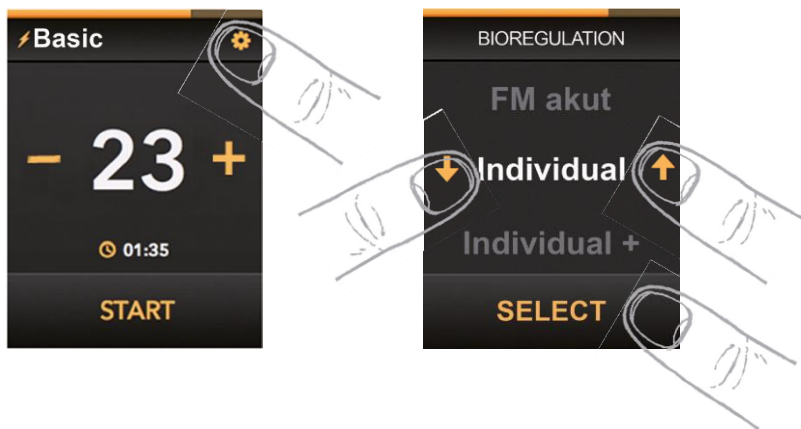


The respective therapy program can be selected with the arrow symbols. The program is then displayed brighter and positioned in the centre. Press "SELECT" to confirm the selection and save this type of therapy. The saved type of therapy is now automatically selected by default if the "START" button is pressed. This function is also available in diagnostic mode. This way, the saved type of therapy can be quickly accessed.

Individual+ therapy program

The Individual+ therapy program includes a dedicated function for setting a special frequency. For this reason, the energy setting is not displayed as usual when selecting Individual+. Instead, step "2. Frequency search" is started.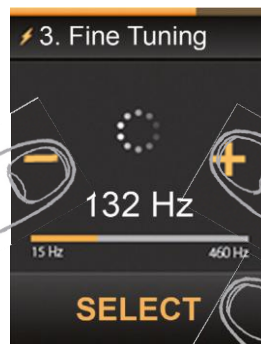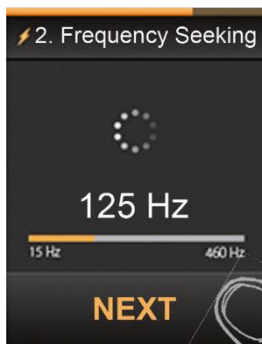

In frequency search, the pulse frequency is slowly and cyclically increased from 5 Hz to 460 Hz. The current frequency is displayed as a number in the centre of the screen and as a running bar. Press "NEXT" to apply the current frequency and to start step "3. Fine settings". In fine settings, the frequency can be increased or decreased in individual steps by pressing the plus or minus buttons. Press "SELECT" to apply the current frequency and to complete the settings for the "Individual+" therapy program.

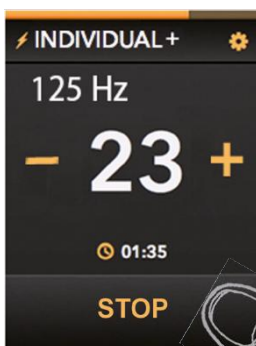
The set frequency is applied in the Individual+ program and displayed. Now, the energy can be increased or decreased as usual. To restart the frequency search, select the Individual+ therapy program again.


Diagnostic mode

The Diagnostic mode works in the same way as the Bioregulation mode. Additional measurement data can be shown on the display with the use of the Diagnostic mode, however. This way, asymmetries can be identified more precisely and objectively. This mode also enables application of a therapy while the dynamics can be displayed according to the acquired values. You will also be shown when the therapy can be ended. Please refer to the seminar content for information on therapy in the Diagnostic mode.

You can activate the Diagnostic mode by pressing the corresponding item in the main menu. A sub-menu is opened in which either a mode with a fixed frequency of 78 Hz ("Freq. 78 Hz") or a freely selectable frequency "Hz Individual" can be selected.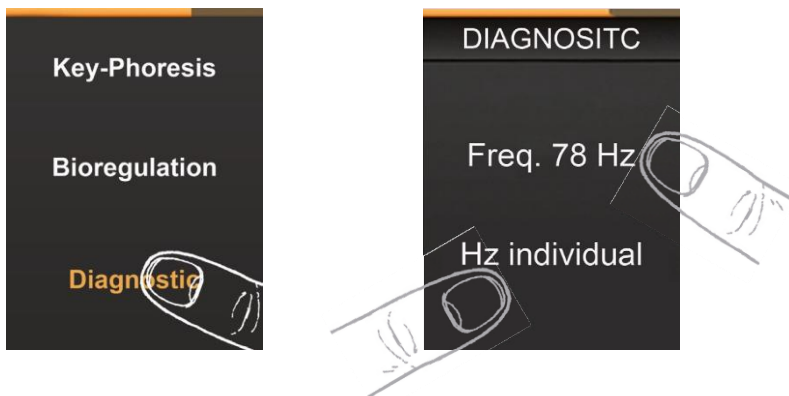

Frequency of 78 Hz diagnostic mode overview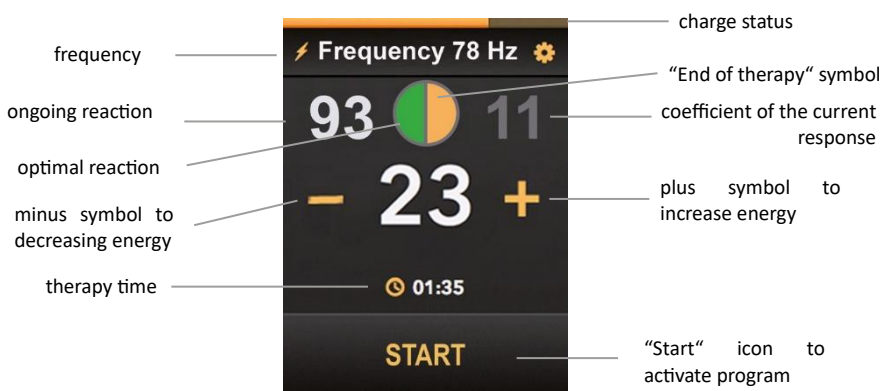


Press the plus or minus buttons to increase or decrease the energy setting. Pressing buttons is confirmed by an acoustic signal.

The energy setting is indicated by the number in the centre. The energy setting can be set between 1 and 99. After the energy setting, the different measurement values are automatically displayed.

The "ongoing reaction" provides information on the responsiveness of the tissue. The coefficient indicates the "zero runs" in the bipolar pulse wave packet.

The two fields in the centre indicate when the maximum efficiency is reached and when no more change in tissue damping is to be expected.

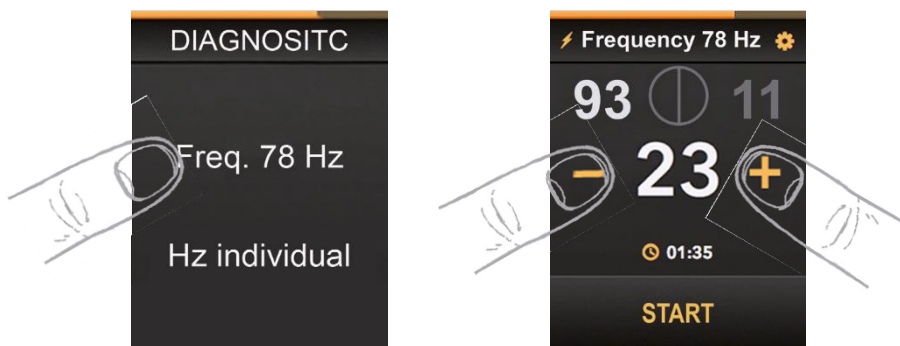
Press "START" to activate the default therapy program (e.g. Triggering). Press "STOP" to return to diagnostic mode.

Hz Individual therapy program in diagnostic mode


The "Hz Individual" therapy program can be accessed via the "Diagnostic" sub-menu to display the "ongoing reaction" and "zero runs" for any frequency.

Under step "1. Energy", the pleasant energy to be kept during frequency search has to be set first. Press "NEXT" to start the actual frequency search function.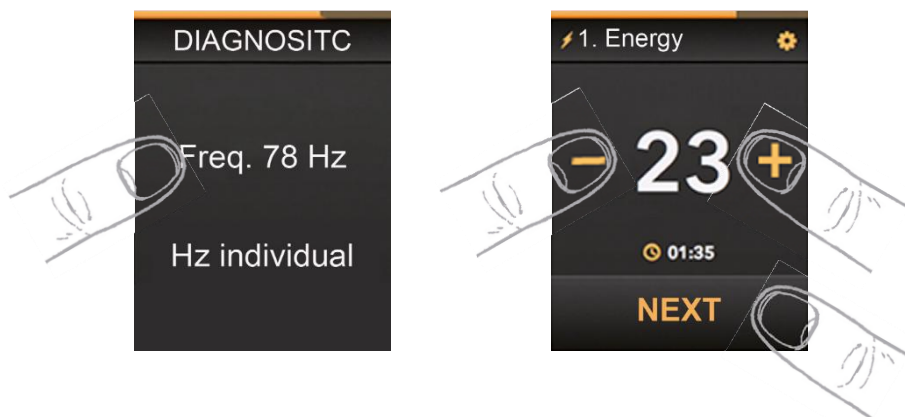


In step "2. Frequency search", the frequency band is cyclically run through from 5 Hz to 460 Hz. The current frequency is displayed as a number and as a bar in the centre of the screen. Press "NEXT" to save the currently displayed frequency and to change to step "3. Fine settings".

In fine settings, the frequency can be increased or decreased in individual steps by pressing the plus or minus buttons. Press "SELECT" to save the selected frequency and to apply it into the individual frequency settings.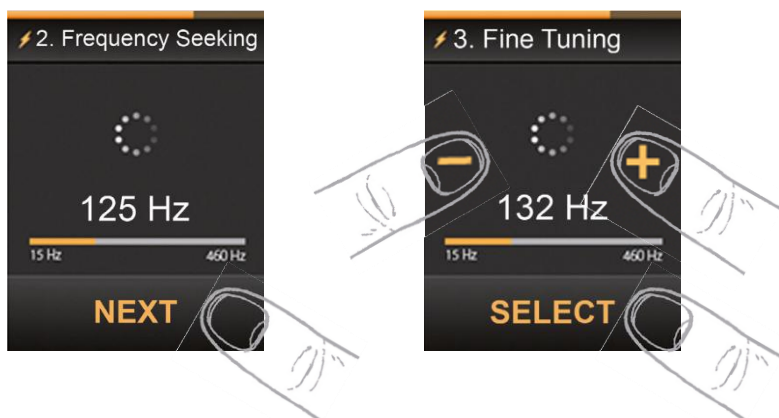

Now, the energy can be increased or decreased as usual. Additionally, the "ongoing reaction" and the "zero runs" are displayed. To restart the frequency search, select the "Hz Individual" therapyprogram again via the diagnostic mode.

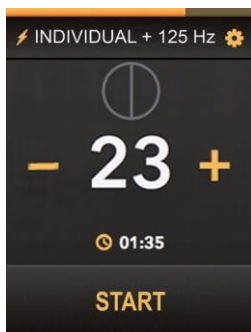Device Settings

After switching on the device, the service menu can be accessed by pressing and **holding** the gear symbol in the top right. The button can be released as soon as the service menu is displayed. Press the arrow buttons at the left and right to select the respective settings from the list by pressing "SELECT". The following settings can be made:

- **Language:** The options English and German are available. The respective option is selected from a list. Press "SELECT" to apply the selected language.
- **Sound:** The sound can be switched on or off. Warning signals such as "low battery" cannot be deactivated. The respective option is selected by means of two buttons for "Sound on" and "Sound off".
- **Brightness:** The maximum brightness of the display can be set in a range of 50 % - 99 %. The respective option is selected from an energy indicator. Press "SELECT" to apply the set value.
- **Touch sensitivity:** The touch display offers seven sensitivity settings. The sensitivity is increased with "+" and decreased with "-". The respective option is selected from a list. Press "SELECT" to apply the set sensitivity setting.
- **Patient recognition:** The following three sensitivity settings can be selected for patient recognition.

High: Quick recognition may cause the device to keep running after use on some patients.

Medium: Recommended setting for optimum recognition for most patients.

Low: Recognition after application with light pressure.

If the "Next" symbol is covered by the symbol with the crossed out hand, briefly press the "Home" button to unlock the display.

Charging the physiokey

The **physiokey** indicates the current battery power during operation with a symbol (here illustrated on the display after starting the default settings):


Before the battery power is too low for ongoing operation of the **physiokey**, an acoustic signal can be heard. If the battery power is low, the **physiokey** cannot be started. In this case, connect it to the charger.

The scope of delivery includes a battery charger with micro USB connector. To charge the **physiokey**, plug the charger in a suitable power socket and connect it with the **physiokey**. The device has to be switched off. It cannot be used during charging. During charging, the screen illustrated below is displayed. After a short time, the display is switched off during charging to save power and to accelerate the charging process. For information on the charging process, press the "Home" button to wake up the display.

The charger socket has to be protected against the ingress of fluids. For this reason, always close the charger socket with the included USB plug after charging. It must be ensured that the charging process is only carried out with clean and dry contacts.

physiokey is charging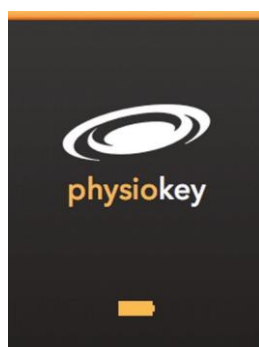

physiokey is fully charged

Battery performance:


The physio**key** battery has been specially adapted to the characteristics of the physio**key**. It is **not removable**. To ensure maximum battery life, please note:

- Should the battery discharge entirely, the physio**key** is broken. Charge the battery if it is running low.
- Switch off the physio**key** if it is not used.
- Charge your device with approved charging accessories only.
- Do not switch the physio**key** on and off in quick succession.


Replacing the Changeable Electrode Head

The electrode on the bottom of the physio**key** is attached to a changeable electrode head. This changeable electrode head can be replaced by specific optional changeable electrode heads considerably expanding the therapy spectrum.


To replace the changeable electrode head, slide back the unlocking slide.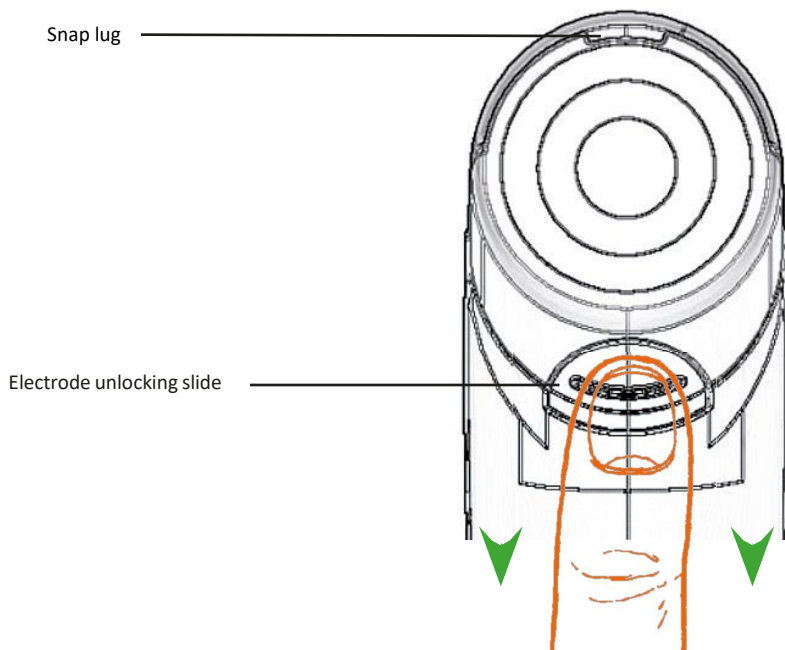

Afterwards, take the changeable electrode head out of its guide by lifting it at the side of the unlocking slide.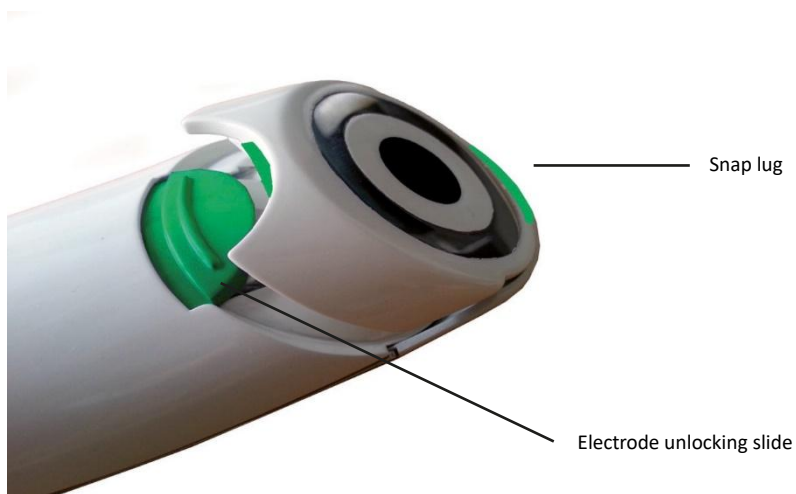

To insert a different changeable electrode head, first slide the recess in the changeable electrode head under the snap lug while tilted. Lower the rear end and engage it with the unlocking slide.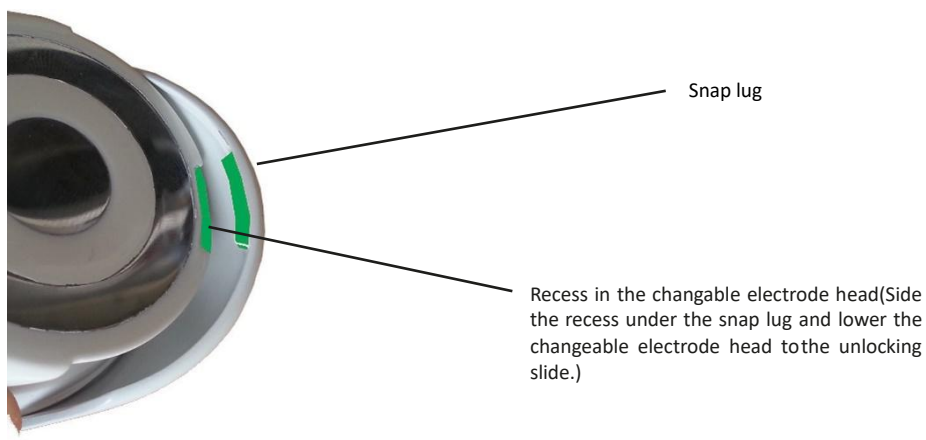

Care, cleaning and storage

Your **physiokey** has been designed so that it is robust and durable. Nevertheless, we strongly recommend you handle your **physiokey** with the same degree of care you would afford to any other technical device. Insufficient or incorrect care and cleaning may void the guarantee and warranty.

The **physiokey and the exchangeable electrode** must be cleaned and disinfected after each therapy session. The base station should also be cleaned/disinfected at regular intervals.

Please proceed as follows:

- Switch off the **physiokey** handpiece after each use.
- Apply cleaning agent and then disinfectant to a microfibre cloth slightly moistened with water or (for disinfection) with a microfibre cloth soaked in disinfectant.
- Gently wipe the device down with the cloth.
- Remove and wipe down the changeable head.
- Clean the device underneath the support surface of the changeable head
- Wipe with a damp cloth which is free of cleaning agent / disinfectant.
- Disconnect the base station from the power pack and clean/disinfect surfaces in the same way.

Isopropyl alcohol is to be used for the disinfection. Alternative procedures are only permitted following a consultation, and if they correspond to the recommendations of the Robert-Koch-Institute (Germany) and/or other national requirements.

Please note that the **physiokey** is not entirely waterproof. For this reason, all parts should only be handled with a piece of cloth that has been moistened with cleaning agent.

Ensure that no humidity penetrates into the interior of the device. Residues of external materials or substances must be removed completely.

Adhesive electrodes: Multiple use of the same adhesive electrodes may only occur on the same person.

Use:

1. Clean and dry the areas to be treated. Remove dirt, grease, creams, etc.
2. Remove 2 of the adhesive electrodes from the base foil and connect the electrode cables via the connector cable
3. Place the adhesive electrodes on the skin at the intended location. Make sure that it lies crease-free.
4. Select the required therapy mode on the Key-series device and set the energy level so that you feel comfortable.
5. At the end of therapy, switch off the Key-series device and disconnect the connector cable from the adhesive electrodes.
6. Carefully remove the adhesive electrodes from the skin and stick them onto the base foil until they are ready for reuse. Do not pull on the cable!
7. Store at room temperature. Protect against excessive moisture.

Store your **physiokey** in the base station and only use the transport case for transport.

Error Messages

The **physiokey** has several built-in self-test and monitoring mechanisms. These monitor the integrated hardware and the function of the firmware.

Error	Likely Cause and Solution
Device shows a blue screen	If the monitoring mechanism recognizes an error, the application is interrupted immediately and a blue screen with an error message is displayed. The device is automatically switched off after 3 seconds with an acoustic interval signal. Press the "Home" button to display the error message. The device can be switched on again with the "Home" button. The self-test after switching on the device must not recognize any error to continue working with the physiokey. If this is not the case, the device can be recalibrated and set to factory settings.
Device cannot be turned on or Device turns itself off automatically	If the device cannot be switched on, the screen remains black or can only be seen briefly, please check the battery. To do so, connect the included charger. If the loading screen is displayed, the battery is charged.
Temperature Error	The use temperature of physiokey is exceeded or fallen below minimum. Wait until the device has cooled down or warmed up. It can be turned on again afterwards.
Lagging touchscreen control	Restart the device and open the settings menu. Select "Touch sensitivity" from the list with the arrow buttons and press "SELECT". Press the down button in the displayed sub-menu. The more plus characters are displayed the higher the sensitivity of the touch-screen controls. Press "SELECT" to apply the new value. After pressing the "Home" button, the device may be used as usual.
Excessively sensitive touchscreen control	Restart the device and open the settings menu. Select "Touch sensitivity" from the list with the arrow buttons and press "SELECT". Press the up button in the displayed sub-menu. The more minus characters are displayed the lower the sensitivity of the touch-screen controls. Press "SELECT" to apply the new value. After pressing the "Home" button, the device may be used as usual.

Error	Likely Cause and Solution
Unintended start of patient recognition	Restart the device and open the settings menu. Select "Patient recognition" from the list with the arrow buttons and press "SELECT". Press the down button in the displayed sub-menu until the "low" setting is displayed. This reduces the sensitivity of patient recognition. Press "SELECT" to apply the new value. After pressing the "Home" button, the device may be used as usual.
Patient recognition not starting	Restart the device and open the settings menu. Select "Patient recognition" from the list with the arrow buttons and press "SELECT". Press the up button in the displayed sub-menu until the "high" setting is displayed. This increases the sensitivity of patient recognition. Press "SELECT" to apply the new value. After pressing the "Home" button, the device may be used as usual.
Input via touchscreen controls not possible and/or Patient recognition does not work with all three setting	If the monitoring mechanism recognizes an error, the application is interrupted immediately and a blue screen with an error message is displayed. Recalibrate the device.
Damage	In case of visible damage at the device, immediately stop using the physiokey. It is recommended to return the device to the manufacturer to have it checked.

Recalibration

CAUTION! Only recalibrate the device when the battery is fully charged!

If a blue screen appears with the words "go to service or reset device", this means that a recalibration is required first. Restart the device. Press the Home button again and hold it for at least 10 seconds until an acoustic signal occurs along with a blue screen. Release the Home button and wait for the device to shut down automatically.

CAUTION! Before switching on the device again, make sure that there is no contact between the electrodes and the patient, you or any other conductive objects!

Switch on the device and wait for the device to calibrate automatically. This may take several minutes.

CAUTION! At this time, there must be no contact between the electrodes and the patient, you or any other conductive objects!

After the calibration has been completed, the settings menu is automatically displayed. Here, you can make the settings you require, such as the language. After pressing the Home button, the device can be used as usual.

Service, repairs and safety inspections (STK)

Contact the manufacturer if

- your **physiokey** does not work properly and the measures in the chapter “Error messages and how to manage them” do not lead to success.
- the battery capacity of your **physiokey** handpiece has fallen below one hour of treatment time (see “Charging the **physiokey** handpiece”). The battery can only be replaced by the manufacturer.

Apart from the tests (see below), which have to be carried out by the user and a battery change when the battery capacity is no longer sufficient and/or a deep discharge has occurred (see p. 22 et seq., “Charging the **physiokey**”), the **physiokey** is maintenance-free for its entire service life.

Type of test	Component to be tested
Visual inspection	Instruction for use available
	Display readable
	Housing parts properly connected
	Housing without damage
	Contacting under exchangeable head undamaged
	Free of residues
	Side electrodes firmly attached
	Charging contact not damaged
Function testing	Display functional
	Touchpanel functional
	Impuls perceptile (Bioregulation)
	Changeable head connection free-running
Functional test optional components	Changeable heads clean and functional

Within the scope of the Medical Device Operator Ordinance (only operators in Germany), a medical device log (book) must also be kept, whereby the form attached by Keytec can be used as a template.

Loss of guarantee and warranty

The guarantee does not apply to wearing parts and consumables. Among others, the following actions lead to loss of guarantee and warranty

- Opening the **physiokey** handpiece or optional accessories.
- Manipulating the internal hardware or firmware of the device.
- Combining the **physiokey** with *non-original-manufacturer* technical equipment.
- Connecting the **physiokey** with a *non-original manufacturer* charger (power pack).
- The improper use of the device.
- Gross negligence in connection with the use of the device (water ingress into the casing, dropping, etc.).

Return and disposal

Since the physio**key** handpiece is equipped with a rechargeable battery, please dispose of the entire device at a collection point which is part of the general take-back and recycling system. Do not open the device. The manufacturer is also obliged to take back the device.

Free collection points and/or acceptance points for used electrical equipment are available in your local area. Their address details should be available from your local council or municipal authority. If your physio**key** contains personal data, you are responsible for deleting such data before you hand it in. For further information, visit www.elektrogesetz.de.

KeyAkademie events

The seminars, workshops, live chats and webinars provided by KeyAkademie serve the purpose of sharing experiences with colleagues and learning about the electrotherapy treatment options with your physio**key**.

For further information on dates and locations, visit www.keyakademie.de.

Attachment: Contact details and technical information

Contact details

Manufacturer:	Keytec GmbH Steppacher Str. 32a 86420 Diedorf Germany
Contact partner Service, repair:	Keytec GmbH Charlottenburger Allee 23a 52068 Aachen Germany
Tel:	+49 241 - 913 88 95 - 0
Email:	info@keyserie.com
Internet:	www.keyserie.com

Explanations of the electrical current output of the different therapy modes

Key-Phoresis mode

Key-Phoresis acid:	Galvanic treatment with iontophoresis characteristic for working in acidic external agents; charge balance from - to +
Key-Phoresis basic:	Galvanic treatment with iontophoresis characteristics for working in basic external agents; charge balance from - to +

Bioregulation mode:

Basic:	Standard frequency – all other parameters inactive
Calming:	High frequency in combination with amplitude modulation
Triggering:	Low frequency with high intensity and amplitude modulation
Relaxation:	Deviating standard frequency with different time interval and intensity
Harmonisation:	Deviating standard frequency, high time interval and low intensity
FM-chronic:	Low frequency modulation
FM-acute:	High frequency modulation
Individual:	Automatic intensity adjustment, variable time interval, frequency interval
Individual+:	Automatic intensity, variable time interval, adjustable frequency
Activation:	Low frequency with damping
Memory	Frequency interval with variable damping

Technical data**Appliance (physiokey handpiece)**

Dimensions:	185 x 51 x 28 mm (L x B x H)
Weight:	180g
Ambient temperature:	Use: 0°C to 30°C Storage: -20°C to 45°C
Ambient humidity:	max. 93% rel. humidity, non-condensing
Ambient pressure:	air 700hPa...1060hPa
Protection class:	IP 42
Power supply:	Li-battery 3.7V nominal 4.2V final charging current
Charging current:	5V +/-0.5V
Max. charging current:	500 mA
Max. charging time:	180 min
Battery charging capacity	800 mAh
Typ operating time	6h (depending on treatment mode and energy level)
Min. charging cycles	500
Energy output:	a) Pulse operation: < 10mA (rms) at 10kOhm(depending on the operating mode) Pulse energy max. < 1 mJ/pulse at 10 kOhmEnergy setting adjustable in 100 settings b) Key -Phoresis DC operation: max. 1.2 mA at 10 kOhm Umax <= 12.5V Current level selectable in 100 steps 0...1mA 50steps [20 µA/step] 1...10 mA 50 steps [180 µA/step]


The device is not disturbed by wireless communication equipment (e.g. mobile phones) nor causes disturbances.

Charger (original accessory) (Type: HNP06-USB-L6)

Input voltage range:	90 ... 264 VAC
Input frequency:	47 ... 63 Hz
Input current:	max. 300 mA (rms) at 90 VAC, 50 mA(rms) at 230 VAC
No-load loss	0.17W
Output voltage:	5V +/- 0.5V
Power output:	6 watts

Symbols and pictograms

The following symbols and icons can be found on your physiokey handpiece: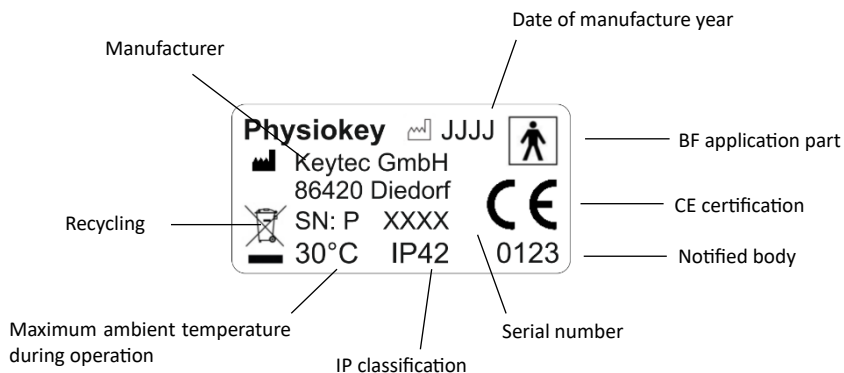

Safety information:


Caution! Instruction for use and all accompanying documents must be observed!

UDI in matrix code:

The matrix code is a machine-readable code that also contains all information about the UDI (except the Basic UDI).

Together with the matrix code, this results in the following image:

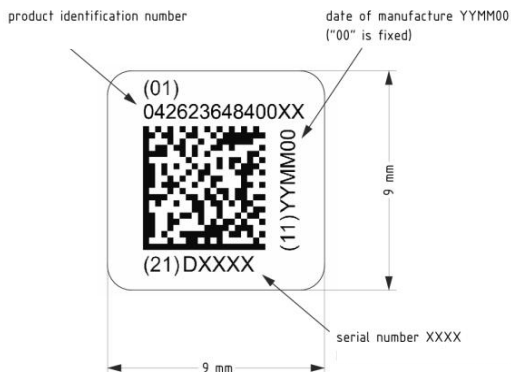Results of EMC tests carried out

Emission tests in accordance with:

Norm:	CISP 11	
Class	B	
Group	1	
Electromagnetic Radiation Disturbance	30 MHz – 1 GHz	passed
Email:	info@keyserie.com	
Internet:	www.keyserie.com	

Immunity Test and their Test-Levels

Electrostatic Discharge (ESD):	Contact Discharge: +8kV	passed
	Air Discharge: +15kV	passed
Electromagnetic Field:	10 V/m	passed
	30 A/m	passed

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