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physio**key** manual  
explanation of handling.

English

# Index

<b>Definition and General Information</b> .....	<b>36</b>
What is the <b>physiokey</b> ? .....	36
Indications .....	36
Fields of Application .....	36
Contraindications and Non-Applications .....	36
Side-Effects .....	37
Safety Instructions .....	37
Briefing and Book of Medical Devices .....	37
<b>Scope of Delivery</b> .....	<b>38</b>
Optional Accessories and their Scope of Application .....	38
<b>Handling</b> .....	<b>39</b>
<b>physiokey</b> Brief Description .....	39
Explanation of the Function of the "Home" Button .....	40
Description Touchscreen Controls .....	40
Inactive Display .....	41
Locking Times of the Display .....	41
<b>physiokey</b> Therapy Modes .....	42
Key-Phoresis Mode .....	43
Bioregulation Mode .....	45
Diagnostic Mode .....	49
Device Settings .....	53
Charging the <b>physiokey</b> .....	54
Replacing the Changeable Electrode Head .....	55
<b>Care and Cleaning</b> .....	<b>57</b>
Error Messages .....	58
Recalibration .....	60
Service and Repair .....	60
Loss of Guarantee and Warranty .....	60
<b>Return and Disposal</b> .....	<b>61</b>
<b>Attachment: Technical Information</b> .....	<b>61</b>
Explanation of the Therapy Modes' Pulse Forms .....	61
Technical Data .....	62
Symbols and Pictograms .....	63

# Definition and General Information

## What is the physiokey?

physiokey is an adaptively regulated electrotherapy device.

The device uses the process of non-invasive electrical nerve stimulation which has been established as a successful therapy for many years, through the electrode's contact with the skin. This process is generally regarded as displaying extremely few side-effects.

physiokey can create bidirectional (in "Bioregulation") and galvanic (in "key-phoresis") forms serving as a base for the following indications and fields of application:

## Indications

The therapeutic aim of the physiokey is to reduce acute and chronic pain, improve blood circulation and perform passive muscle stimulation for the following fields of application:

## Fields of Application

- Nervous system
- Musculoskeletal system
- Digestive tract
- Urogenital area

The recommended treatment duration for the devices ranges from a few minutes to a maximum of 1 hour. The treatment is usually performed by moving the device on the skin.

physiokey is a medical product. The field of application is both in trained, professional therapy, as well as home-use of medical laypersons.

## Contraindications and Non-Applications

In the following cases, physiokey may not be used or only in consultation with a doctor:

- Use prohibited on injured skin (e.g. open wounds and wounds which have not yet completely healed)
- Use prohibited in body orifices
- Use prohibited on carriers of active controlled implants (e.g. cardiac pacemakers)
- Use prohibited in the area of the uterus of pregnant women
- Use prohibited above tumours
- Use prohibited on patients with cardiac arrhythmia
- Use without consultation with a doctor prohibited on patients with malign diseases

## Side-Effects

The **physiokey** shows extremely few side-effects. The following can, however, be regarded as side-effects:

Symptom intensification: Symptoms may intensify during the treatment period since the body is "dynamised". In individual cases, this may be accompanied by stronger pain, muscle ache and other reactions, but they generally cease quickly. Should they last longer, you should interrupt the treatment for a few days and, if necessary, consult a doctor.

Like all metals, the steel of the electrodes may contain traces of nickel. Patients with pronounced nickel allergy may therefore experience skin irritation.

## Safety Instructions

- Read the manual (this document) before you use the device for the first time.
- **physiokey** must be kept out of reach of children and stored accordingly. Numerous dangers might arise if the devices gets into children's hands.
- Use the **physiokey** in continuous operation only at ambient temperatures of up to 30 °C max.
- Do not expose the **physiokey** to temperatures above 40 °C and protect it against direct sunlight.
- Do not submerge the **physiokey** in water or hold it under flowing water.
- Do not dispose of the device in household waste, but according to special waste regulations.
- Do not open the device.
- Do not use it near inflammable substances.
- Only use the **physiokey** for the intended purpose.
- Only charge the device with the delivered charger.

## Briefing and Book of Medical Devices

You can receive a briefing into the device use

- by visiting seminars of **KeyAkademie** (German-only; details: [www.keyakademie.de](http://www.keyakademie.de); fees apply)
- by in-house seminars of **KeyAkademie** (German only; fees apply)
- through telephone support of a speaker of **KeyAkademie** (German only; free of charge)
- through your medical product consultant (free of charge)

Safety-related checks have to be performed by professional staff according to German law (if the device is operated in Germany). If requested, the manufacturer can support through providing a protocol and fee-based service.

# Scope of Delivery

The scope of delivery of the physiokey includes:

- Hand-held device
- Standard changeable electrode head
- Charging device with mini USB connector
- user manual
- Base station

## Optional Accessories and their Scope of Application

Highpro. Ideal for cross-dermatome treatment and therapy of meridians.

Trigger. Treatment of small-scale structures and muscle strands, tendons, trigger points.

Matrix. Specialised for fascia therapy and treatment of connective tissue.

Comb. For treatment of hairy areas.

Connector cable. Connection of various electrodes for long-term therapy.


Faszien-Flow. For treatment of fascia and trigger points, face and **key**-phoresis.

Tabletop base station. Secure storage with charging function.

## Handling

### physiokey Brief Description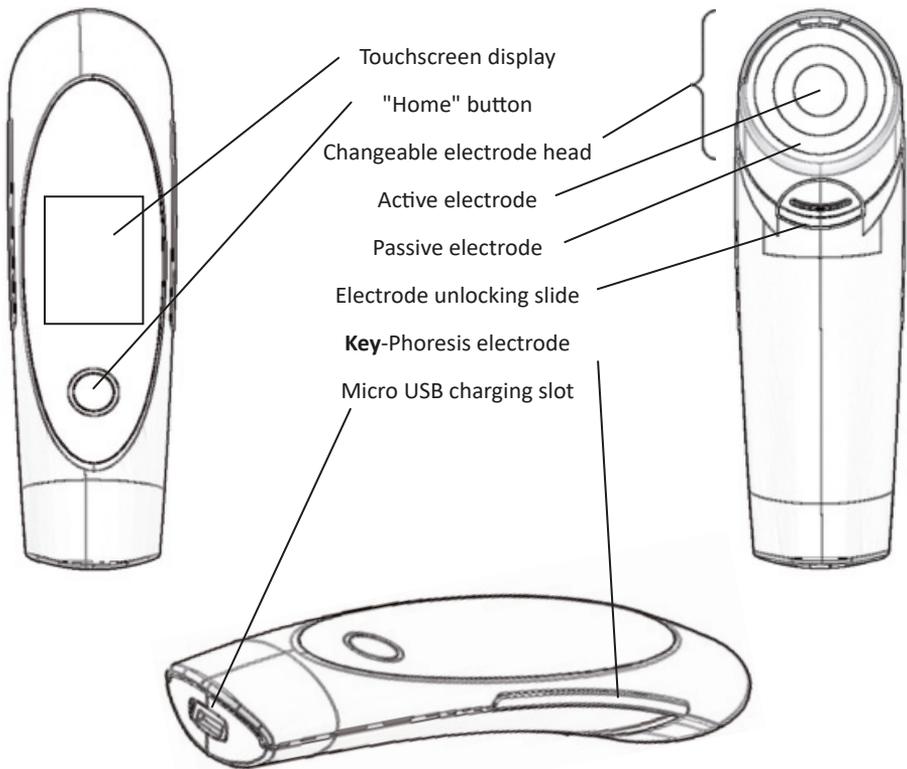

## Explanation of the Function of the "Home" Button

The "Home" button is the only physical pushbutton on the physiokey and offers 4 functions:


1. **On/Off:** With the "Home" button, the device is switched on. To do so, press the button for two seconds. By pressing the button again for two seconds, the device is switched off again. Please wait at least 15 seconds before switching on the device again.
2. **Back to Main Menu:** the second function of the "Home" button is to return to the main menu by briefly pressing the button while the touchscreen is active.
3. **Reactivating of the Display:** If the touchscreen controls are not active, they can be reactivated by briefly pressing the "Home" button.
4. **Reset/Recalibration,** to be executed if a bluescreen is shown. See chapter troubleshooting.

## Description Touchscreen Controls

The display of the physiokey features touchscreen controls. If the energy setting or the therapy mode is to be changed, simply press the respective symbol or button on the display. In this description, the phrase "press" is often used regarding symbols/buttons. However, light touching is often sufficient. In general, touching the button directly triggers the intended function and is confirmed by an acoustic signal. In a list, however, functions are selected first with the arrow symbols and then started by another selection symbol (see description: Device Settings).

PLEASE NOTE: Handling the touchscreen is a matter of practice. After a few applications, the functions of the touchscreen will be familiar.

PLEASE NOTE: The touch sensitivity can be set in the service menu (see chapter "Device Settings").

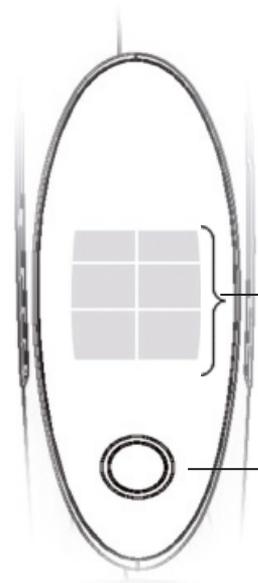The **physiokey** is controlled via the specially shaped control ellipse. This control ellipse is designed in such a way that accidental changes of the selected parameters can almost be excluded. The following controls are available:

Touchscreen controls and display  
(see section: Touchscreen controls)

"Home" button with on/off function

## Inactive Display

To prevent accidental activation of the touchscreen, the touchscreen functions are subsequently locked if they are not used for a specific period of time. If one section of the touchscreen controls is inactive, the following symbol is displayed at the bottom of the screen:

Additionally, the locked buttons turn white.

Briefly press the "Home" button once to reactivate the touchscreen display. If the display is activated, press the "Home" button once to return to the main menu.


## Locking Times of the Display

Already after 4 seconds, only the plus and minus buttons remain active to adjust the energy setting. If the energy setting is not changed for 9 seconds, all touchscreen controls are locked and the display brightness is reduced. Press the "Home" button to fully reactivate the touchscreen controls.

## physiokey Therapy Modes

Make sure that the **physiokey** is fully charged and use it in accordance with the indications, fields of application and contra indications.

Press the "Home" button for approx. two seconds to switch on the **physiokey**. The device is now booted. This is indicated by the following display: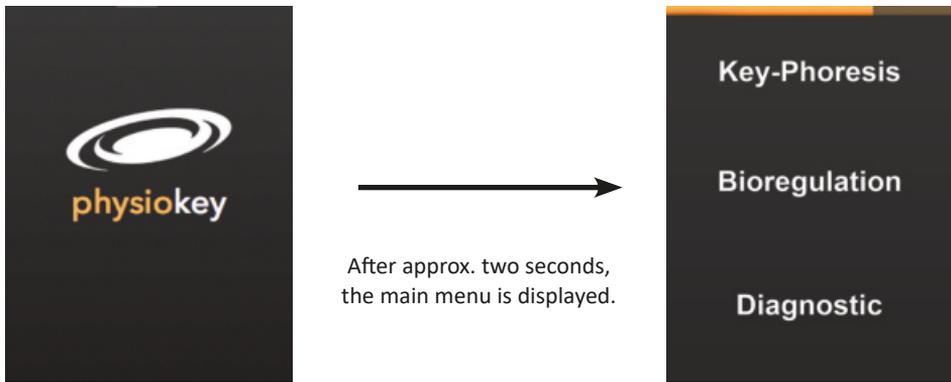

Once the **physiokey** has been turned on, the main menu will always show. From this menu, you have three choices available. Each choice results in a significantly different application/therapy technique. How to use the modes in detail is explained in training manuals or by attending training seminars.

For further information, please refer to [www.keyakademie.de](http://www.keyakademie.de).


## Key-Phoresis Mode

**Key-Phoresis** is a special form of application using the characteristics of direct current. This enables the application of the galvanic treatment, which uses a low direct current. The physio**key** electrode is applied on the skin and the hand-held electrode (on the side) is required as counter electrode. It is naturally touched when using the device and the contact ensures that the electrode effect is applied. To close the circuit, contact with the ideally slightly moistened skin (also the hand) is required.

**Key-Phoresis is generally not felt.**  
**Nevertheless, the desired effect is achieved.**

During **key-phoresis**, the device can also be handled by the patients themselves.

**Key-Phoresis** is activated by pressing the respective button.


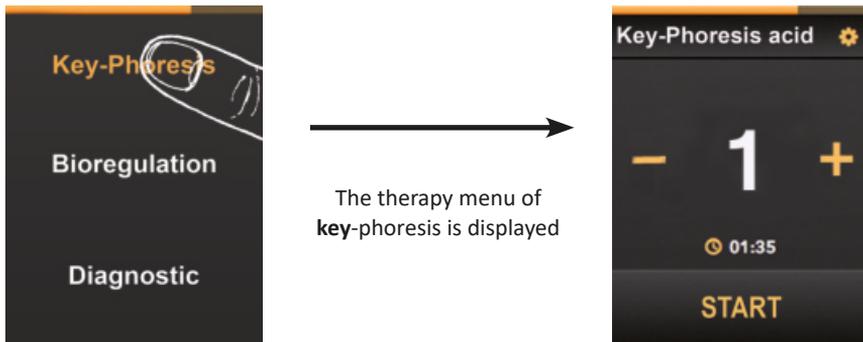
The therapy menu of  
**key-phoresis** is displayed



### Key-Phoresis mode overview

To carry out **key-phoresis**, increase the energy setting by pressing the plus button. Press the minus button to decrease the energy setting. The energy setting is indicated by the number in the centre. The energy setting can be set between 1 and 99. An energy setting that is barely noticeable or not noticeable should be selected. After selecting the energy setting, press "START". To stop the application, press "STOP".

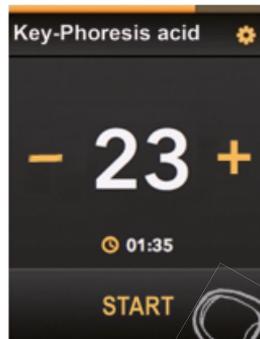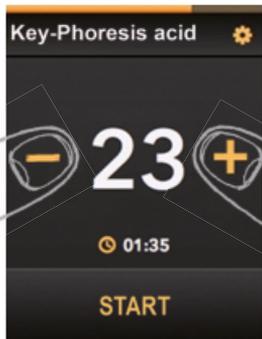
To change the setting from basic to acid, press the gear symbol. Another sub-menu is displayed. Now, the two options are available and can be set by pressing the respective button. The last active setting is indicated by a brighter button.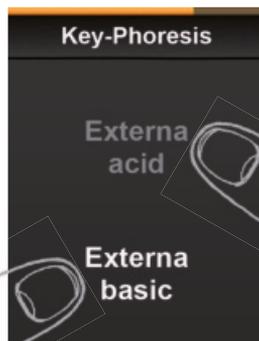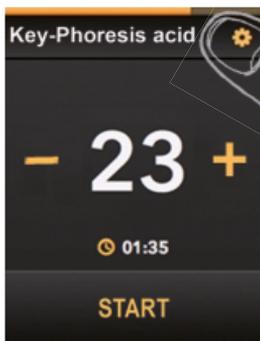

## Bioregulation Mode

The bioregulation mode is selected by pressing the respective button in the main menu.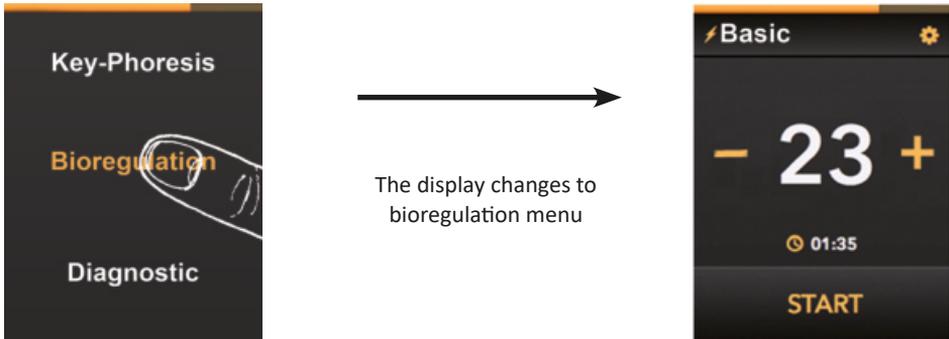

### Bioregulation mode overview



If the lightning symbol is displayed in front of a therapy program, the output energy is applied at the electrodes.

Press the plus or minus buttons to increase or decrease the energy setting. Pressing buttons is confirmed by an acoustic signal. The energy setting is indicated by the number in the centre. The energy setting can be set between 1 and 99. In most cases, an energy level pleasant for the patient should be selected. These settings may also be applied for basic treatment. After switching on the **physiokey**, the therapy program in bioregulation mode is always set to "basic" with an energy setting of 1.


Press "START" to activate a previously saved therapy program. This is indicated by the description above (e.g. Triggering). Press "STOP" to return to basic settings.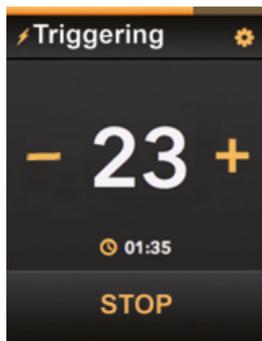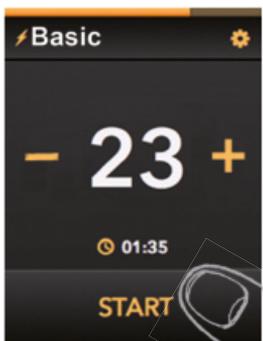

## Changing the saved therapy program

The physiokey offers a large selection of different therapy programs. This way, the device offers maximum flexibility. The different therapy programs are labelled according to the effect to be achieved. For the programs to be selected in specific cases, please refer to the seminar documentation. The selection is made by pressing the gear symbol to open an additional sub-menu. The selection is made by pressing the gear symbol to open an additional sub-menu.

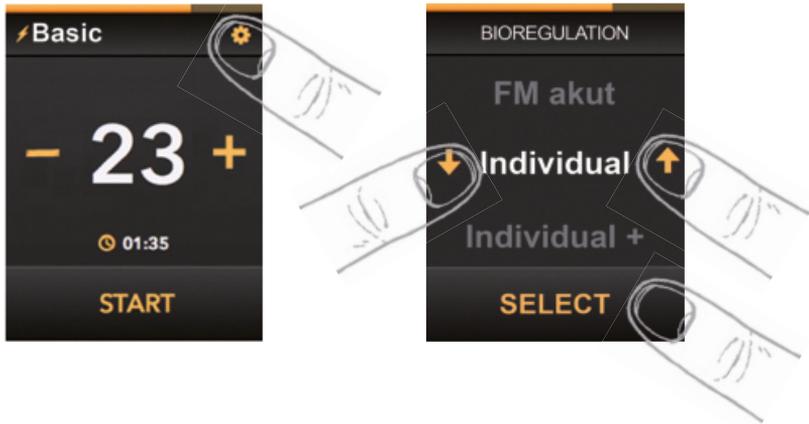The respective therapy program can be selected with the arrow symbols. The program is then displayed brighter and positioned in the centre. Press "SELECT" to confirm the selection and save this type of therapy. The saved type of therapy is now automatically selected by default if the "START" button is pressed. This function is also available in diagnostic mode. This way, the saved type of therapy can be quickly accessed.

## Individual+ therapy program

The Individual+ therapy program includes a dedicated function for setting a special frequency. For this reason, the energy setting is not displayed as usual when selecting Individual+. Instead, step "2. Frequency search" is started.

In frequency search, the pulse frequency is slowly and cyclically increased from 5 Hz to 460 Hz. The current frequency is displayed as a number in the centre of the screen and as a running bar. Press "NEXT" to apply the current frequency and to start step "3. Fine settings". In fine settings, the frequency can be increased or decreased in individual steps by pressing the plus or minus buttons. Press "SELECT" to apply the current frequency and to complete the settings for the "Individual+" therapy program.


The set frequency is applied in the Individual+ program and displayed. Now, the energy can be increased or decreased as usual. To restart the frequency search, select the Individual+ therapy program again.


## Diagnostic Mode

The diagnostic mode enables access to and analysis of measurement data. This way, "asymmetries" can be identified more precisely and objectively. This mode also enables application of a therapy while the dynamics can be displayed according to the acquired values. The time after which the therapy can be ended is also indicated. For therapy in diagnostic mode, please refer to the seminar documentation.

The diagnostic mode is selected by pressing the respective button in the main menu. A sub-menu is opened in which either a mode with a fixed frequency of 78 Hz ("Freq. 78 Hz") or a freely selectable frequency "Hz Individual" can be selected.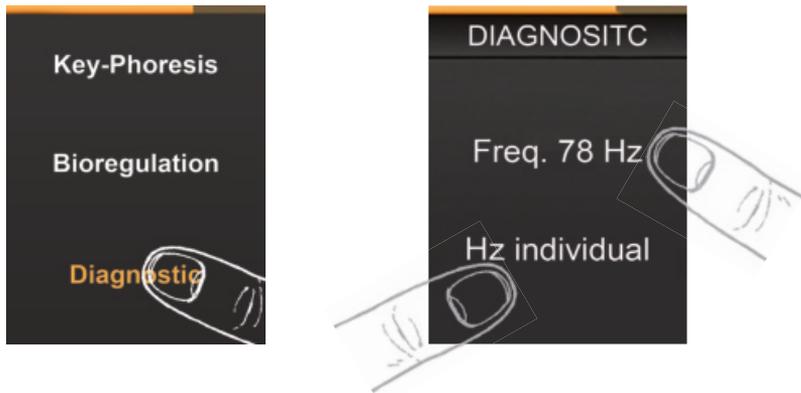

### Frequency of 78 Hz diagnostic mode overview

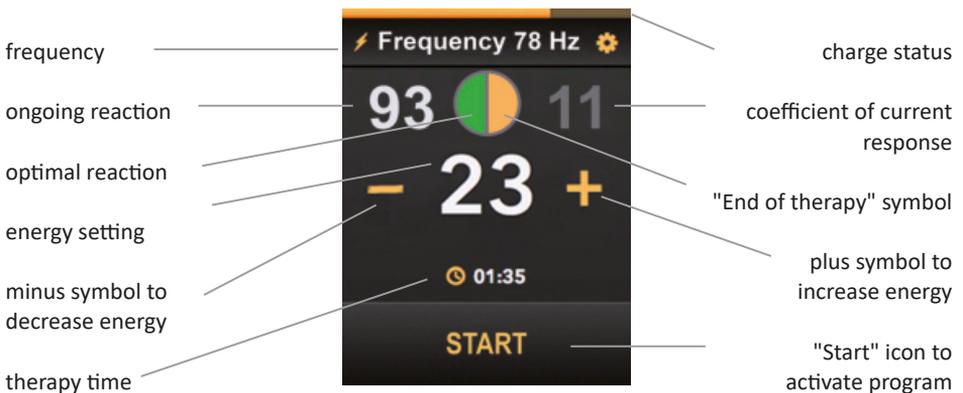
Press the plus or minus buttons to increase or decrease the energy setting. Pressing buttons is confirmed by an acoustic signal.


The energy setting is indicated by the number in the centre. The energy setting can be set between 1 and 99. After the energy setting, the different measurement values are automatically displayed.

The "ongoing reaction" provides information on the responsiveness of the tissue. The coefficient indicates the "zero runs" in the bipolar pulse wave packet.

The two fields in the centre indicate when the maximum efficiency is reached and when no more change in tissue damping is to be expected.

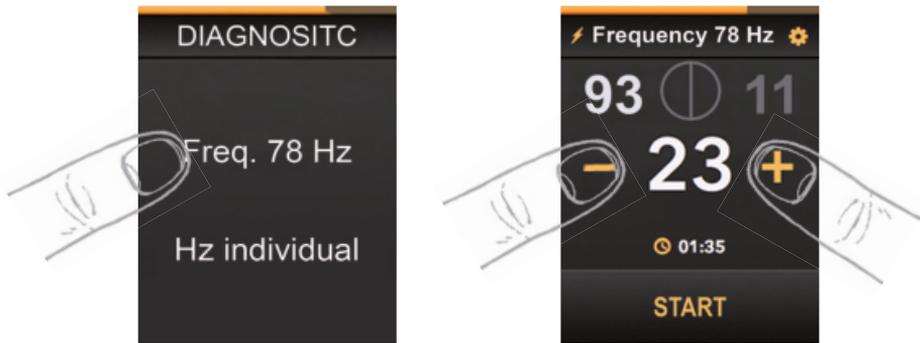
Press "START" to activate the default therapy program (e.g. Triggering) (see page 14). Press "STOP" to return to diagnostic mode.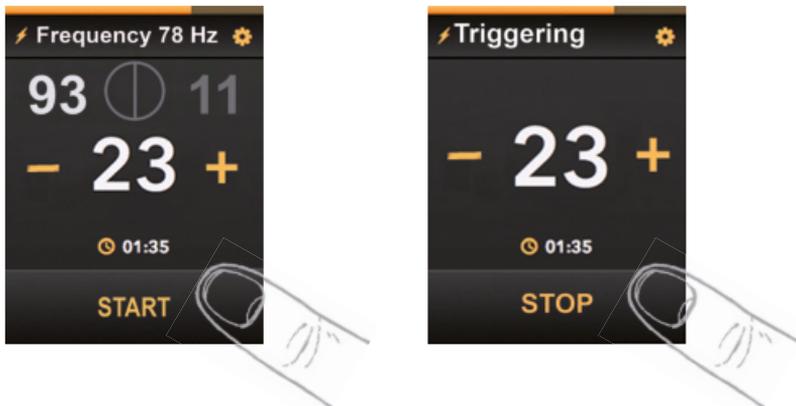


#### Hz Individual therapy program in diagnostic mode

The "Hz Individual" therapy program can be accessed via the "Diagnostic" sub-menu to display the "ongoing reaction" and "zero runs" for any frequency. Under step "1. Energy", the pleasant energy to be kept during frequency search has to be set first. Press "NEXT" to start the actual frequency search function.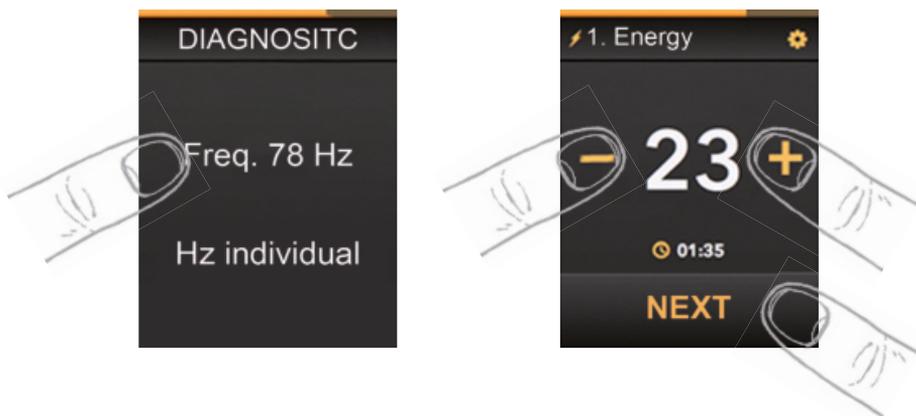


In step "2. Frequency search", the frequency band is cyclically run through from 5 Hz to 460 Hz. The current frequency is displayed as a number and as a bar in the centre of the screen. Press "NEXT" to save the currently displayed frequency and to change to step "3. Fine settings".

In fine settings, the frequency can be increased or decreased in individual steps by pressing the plus or minus buttons. Press "SELECT" to save the selected frequency and to apply it into the individual frequency settings.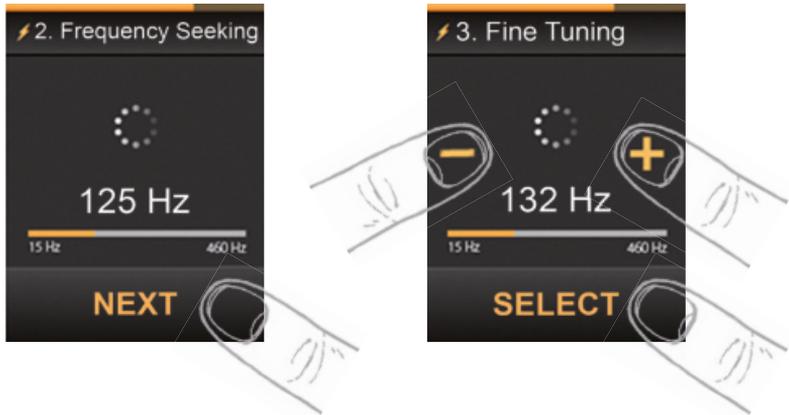

Now, the energy can be increased or decreased as usual. Additionally, the "ongoing reaction" and the "zero runs" are displayed. To restart the frequency search, select the "Hz Individual" therapy program again via the diagnostic mode.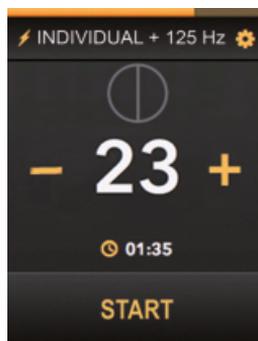

## Device Settings

After switching on the device, the service menu can be accessed by pressing and **holding** the gear symbol in the top right. The button can be released as soon as the service menu is displayed. Press the arrow buttons at the left and right to select the respective settings from the list by pressing "SELECT". The following settings can be made:

- **Language:** The options English and German are available. The respective option is selected from a list. Press "SELECT" to apply the selected language.
- **Sound:** The sound can be switched on or off. Warning signals such as "low battery" cannot be deactivated. The respective option is selected by means of two buttons for "Sound on" and "Sound off".
- **Brightness:** The maximum brightness of the display can be set in a range of 50 % - 99 %. The respective option is selected from an energy indicator. Press "SELECT" to apply the set value.
- **Touch sensitivity:** The touch display offers seven sensitivity settings. The sensitivity is increased with "+" and decreased with "-". The respective option is selected from a list. Press "SELECT" to apply the set sensitivity setting.
- **Patient recognition:** The following three sensitivity settings can be selected for patient recognition.  
**High:** Quick recognition may cause the device to keep running after use on some patients.  
**Medium:** Recommended setting for optimum recognition for most patients.  
**Low:** Recognition after application with light pressure.


If the "Next" symbol is covered by the symbol with the crossed out hand, briefly press the "Home" button to unlock the display.

## Charging the physiokey

The **physiokey** indicates the current battery power during operation with a symbol (here illustrated on the display after starting the default settings):

Battery power: \_\_\_\_\_


Bright = Residual power  
Dark = Consumed power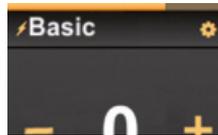


Before the battery power is too low for ongoing operation of the **physiokey**, an acoustic signal can be heard. If the battery power is low, the **physiokey** cannot be started. In this case, connect it to the charger.

The scope of delivery includes a battery charger with micro USB connector. To charge the **physiokey**, plug the charger in a suitable power socket and connect it with the **physiokey**. The device has to be switched off. It cannot be used during charging. During charging, the screen illustrated below is displayed. After a short time, the display is switched off during charging to save power and to accelerate the charging process. For information on the charging process, press the "Home" button to wake up the display.

The charger socket has to be protected against the ingress of fluids. For this reason, always close the charger socket with the included USB plug after charging.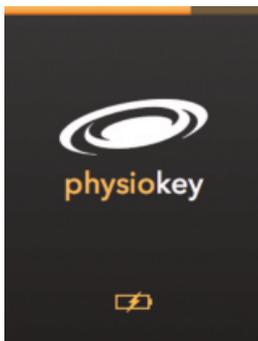

physiokey is charging

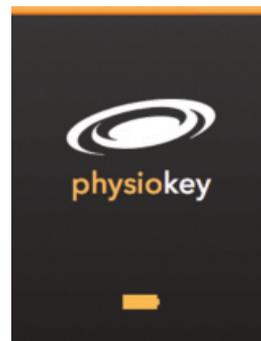physiokey is fully charged

## Battery performance:

The physiokey battery has been specially adapted to the characteristics of the physiokey. It is **not removable**. To ensure maximum battery life, please note:


- Should the battery discharge entirely, the physiokey is broken. Charge the battery if it is running low.
- Switch off the physiokey if it is not used.
- Charge your device with approved charging accessories only.
- Do not switch the physiokey on and off in quick succession.

The battery should last for at least 1 hour depending on the applied therapy modes and the energy setting. If the minimum operating time is clearly below this value, please contact the technical service to replace battery (fees apply).


## Replacing the Changeable Electrode Head

The electrode on the bottom of the physiokey is attached to a changeable electrode head. This changeable electrode head can be replaced by specific optional changeable electrode heads considerably expanding the therapy spectrum.


To replace the changeable electrode head, slide back the unlocking slide.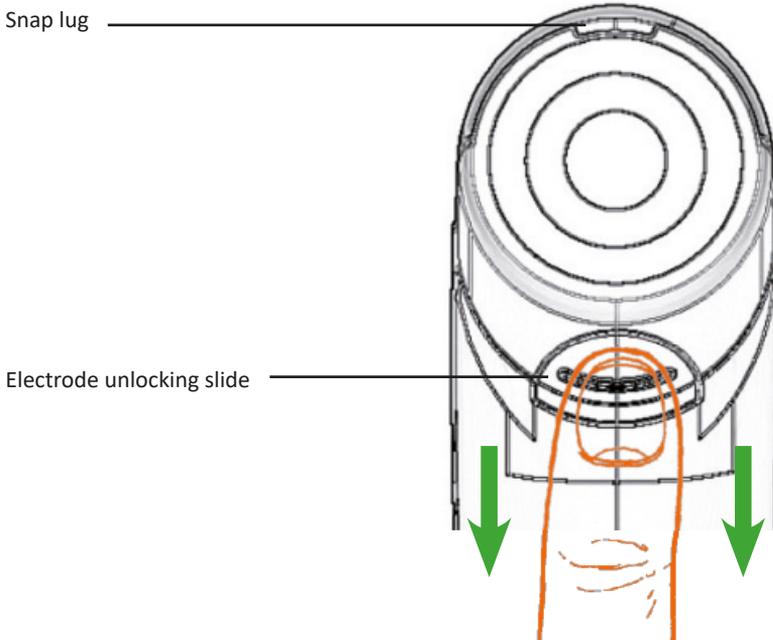

Afterwards, take the changeable electrode head out of its guide by lifting it at the side of the unlocking slide.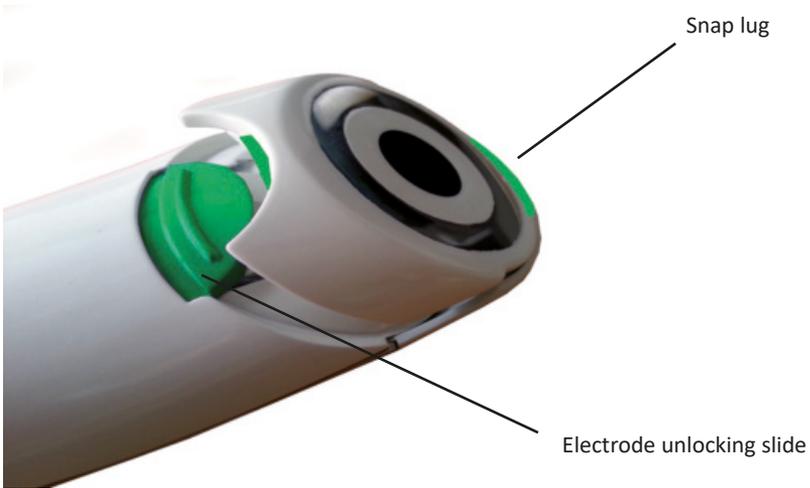

To insert a different changeable electrode head, first slide the recess in the changeable electrode head under the snap lug while tilted. Lower the rear end and engage it with the unlocking slide.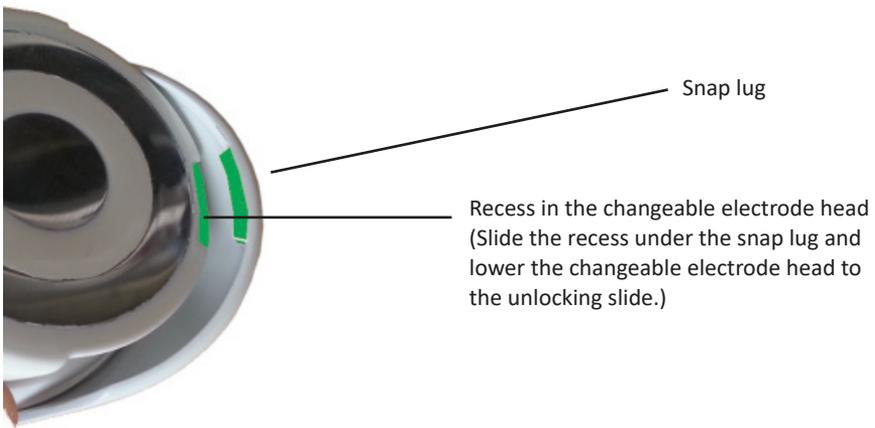

# Care and Cleaning

The **physiokey** design is robust and durable. Nevertheless, we strongly recommend treating the **physiokey** with great care like any other technical device. Insufficient or incorrect care and cleaning may void the warranty. The same applies to gross negligence.

It is recommended to clean and disinfect both the **device and the changable electrode head** part after each therapy treatment and before the beginning of a new therapy treatment with different patients.

## Please proceed as follows:

- Switch off the **physiokey** after each use.
- Recommendation: Clean the **physiokey** after each use.
- Apply cleaning agent on a moist soft piece of cloth.
- Wipe off the device.
- Remove and wipe off the changeable electrode head.
- Wipe afterwards with a moist cleaning cloth without cleaning agent.

Isopropyl alcohol is to be used for disinfection. Alternative procedures must align with national requirements and the Robert-Koch-Institute's recommendations (Germany). Avoid the use of ethanol, especially on the parts (display) made from acrylic!

Please note that the **physiokey** is not waterproof. For this reason, all parts should only be handled with a piece of cloth moistened with cleaning agent. If required, the device can be cleaned extensively with the following means:

- Soap solution (dilution)
- Grease detergent (dilution)

Ensure that no humidity penetrates the interior of the device.

## Error Messages

The **physiokey** has several built-in self-test and monitoring mechanisms. These monitor the integrated hardware and the function of the firmware.

Error	Likely Cause and Solution
Device shows a blue screen	<p>If the monitoring mechanism recognizes an error, the application is interrupted immediately and a blue screen with an error message is displayed.</p> <p>The device is automatically switched off after 3 seconds with an acoustic interval signal. Press the "Home" button to display the error message.</p> <p>The device can be switched on again with the "Home" button. The self-test after switching on the device must not recognize any errors to continue working with the <b>physiokey</b>. If this is not the case, the device can be recalibrated and set to factory settings.</p>
Device cannot be turned on <i>or</i> Device turns itself off automatically	<p>If the device cannot be switched on, the screen remains black or can only be seen briefly, please check the battery. To do so, connect the included charger. If the loading screen is displayed, the battery is charged (refer to page 54).</p>
Temperature Error	<p>The use temperature of <b>physiokey</b> is exceeded or fallen below minimum. Wait until the device has cooled down or warmed up. It can be turned on again afterwards.</p>
Lagging touchscreen control	<p>Restart the device and open the settings menu. Select "Touch sensitivity" from the list with the arrow buttons and press "SELECT". Press the down button in the displayed sub-menu. The more plus characters are displayed the higher the sensitivity of the touchscreen controls. Press "SELECT" to apply the new value. After pressing the "Home" button, the device may be used as usual.</p>
Excessively sensitive touchscreen control	<p>Restart the device and open the settings menu. Select "Touch sensitivity" from the list with the arrow buttons and press "SELECT". Press the up button in the displayed sub-menu. The more minus characters are displayed the lower the sensitivity of the touchscreen controls. Press "SELECT" to apply the new value. After pressing the "Home" button, the device may be used as usual.</p>

Error	Likely Cause and Solution
Unintended start of patient recognition	Restart the device and open the settings menu. Select "Patient recognition" from the list with the arrow buttons and press "SELECT". Press the down button in the displayed sub-menu until the "low" setting is displayed. This reduces the sensitivity of patient recognition. Press "SELECT" to apply the new value. After pressing the "Home" button, the device may be used as usual.
Patient recognition not starting	Restart the device and open the settings menu. Select "Patient recognition" from the list with the arrow buttons and press "SELECT". Press the up button in the displayed sub-menu until the "high" setting is displayed. This increases the sensitivity of patient recognition. Press "SELECT" to apply the new value. After pressing the "Home" button, the device may be used as usual.
Input via touchscreen controls not possible <i>and/or</i> patient recognition does not work with all three settings	If the monitoring mechanism recognizes an error, the application is interrupted immediately and a blue screen with an error message is displayed. Recalibrate the device.
Damage	In case of visible damage at the device, immediately stop using the <b>physiokey</b> . It is recommended to return the device to the manufacturer to have it checked.

## Recalibration

PLEASE NOTE: Recalibrate the device only with a fully charged battery! A blue screen with an error message "go to service or reset device" is displayed.

Restart the device. Press the "Home" button again and hold it for 10 seconds until a "blue" screen is displayed with an acoustic signal. Release the "Home" button and wait for the device to shut down automatically.

**CAUTION!!!** Before switching on the device again, make sure that there is no contact between the electrodes and the patient, you or any other conductive objects.

Switch on the device and wait for the device to calibrate automatically. This may take up to one minute.

**CAUTION!!!** In this time, there must be no contact between the electrodes and the patient, you or any other conductive objects.

After calibration, the settings menu is automatically displayed. Here, settings like the language may be changed as usual. After pressing the "Home" button, the device may be used as usual. The self-test after switching on the device must not recognize any errors to continue working with the **physiokey**. If this is not the case, the device is switched off again. In this case, the device should be sent to the technical service.

## Service and Repair

The manufacturer is only considered to be responsible for safe operation of the **physiokey** if repairs, updates and other modifications are carried out by the manufacturer himself and the device is applied in accordance with its intended use.

The battery may only be changed by the manufacturer.

## Loss of Guarantee and Warranty

The guarantee excludes wear parts and consumables. The following actions lead to loss of guarantee and warranty:

- Attempts to open the **physiokey**.
- Attempts to manipulate the internal hardware or firmware of the device.
- Combination of the **physiokey** with other technical means, e.g. foreign electrodes.
- Charging with a foreign, non-original charging cable.
- Unintended use of the device.
- Gross negligence in connection with the use of the device (water ingress into the housing, fall, etc.).

# Return and Disposal

Batteries must not be disposed of in household waste. The return of empty batteries is legally required.

As the physiokey is equipped with a battery, please dispose of the entire device according to applicable waste regulations. Do not open the device. The manufacturer is obliged to take back the device.

## Attachment: Technical Information

### Explanation of the Therapy Modes' Pulse Forms

#### Bioregulation mode:

Basic:	Standard frequency – all other parameters inactive
Calming:	High frequency in combination with amplitude modulation
Triggering:	Low frequency with high intensity and amplitude modulation
Relaxation:	Deviating standard frequency with different time interval and intensity
Harmonisation:	Deviating standard frequency, high time interval and low intensity
FM-chronic:	Low frequency modulation
FM-acute:	High frequency modulation
Individual:	Automatic intensity adjustment, variable time interval, frequency interval
Individual+:	Automatic intensity, variable time interval, adjustable frequency
Activation:	Low frequency with damping
Memory:	Frequency interval with variable damping

#### Key-Phoresis mode:

Key-Phoresis acid:	Galvanic treatment with iontophoresis characteristic for working in acidic external agents; charge balance from - to +
Key-Phoresis basic:	Galvanic treatment with iontophoresis characteristic for working in basic external agents; charge balance from - to +

# Technical Data

physiokey hand-held device:

Manufacturer:	Keytec GmbH Steppacherstr. 32a 86420 Diedorf Germany
Contact:	Keytec GmbH Münchweg 2a 50374 Erftstadt
Phone:	+49-2235/988 6111
Fax:	+49-6195/969 444 7
E-mail:	info@keyserie.com
Internet:	www.keyserie.com
Dimensions:	185 x 51 x 28 mm (L x W x H)
Weight:	180g
Ambient temperature:	-25°C ... 50°C transport and storage 0°C ... 30°C operation
Ambient humidity:	max. 93% rel. humidity, non-condensing
Ambient air pressure:	700hPa ... 1060hPa
Protection class:	IP 42
Charging voltage:	5V +/- 0.5V
Max. charging current:	500 mA
Max. charging time:	180 min
Battery capacity:	800 mAh
Typ. operating time:	6h (depending on the treatment mode and energy setting)
Min. charging cycles:	500
Output value:	a) Pulse operation: < 10mA (rms) at 10kOhm (depending on the operating mode) Pulse energy max. < 1 mJ/pulse at 10 kOhm Energy setting adjustable in 100 settings  b) <b>Key-Phoresis</b> DC operation: max. 1.2 mA at 10 kOhm U <sub>max</sub> <= 12.5V Power setting adjustable in 100 settings 0 ... 1mA 50 settings [20 µA/setting] 1 ... 10mA 50 settings [180 µA/setting]

The device is not disturbed by wireless communication equipment (e.g. mobile phones) nor causes disturbances.


**Charger (original accessory) (Type: HNP06-USB-L6)**

Input voltage range:	90 ... 264 VAC
Input frequency:	47 ... 63 Hz
Input current:	max. 300 mA (rms) at 90 VAC, 50 mA(rms) at 230 VAC
No-load loss:	0.17W
Output voltage:	5V +/- 0.5V
Power output:	6 watts

## Symbols and Pictograms

The physiokey is labelled with the following symbols and pictograms:

Nameplate:

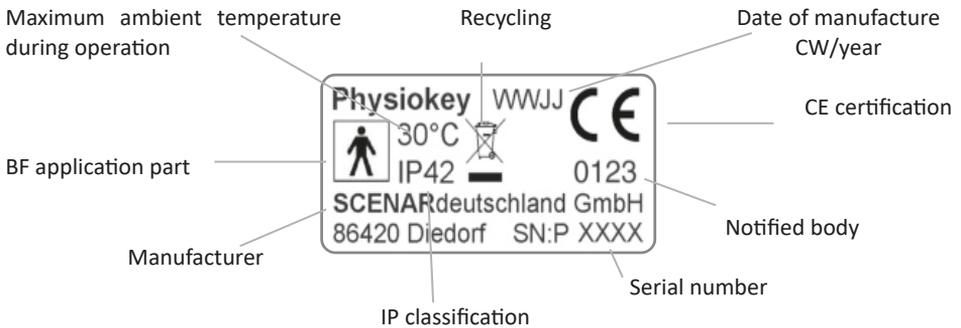Safety information:

**Caution!** Instructions for use and all accompanying documents must be observed!

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