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Class: B

Group: 1

Electromagnetic Radiation Disturbance      30 MHz – 1 GHz      passed

### Immunity Tests and their Test- Levels

Electrostatic Discharge (ESD)      Contact Discharge:      + 8 kV      passed

Air Discharge:      +15 kV      passed

Electromagnetic Field      10 V/m      passed

Power frequency magnetic field      30 A/m      passed

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English

# Index

<b>Definition and General Information</b> .....	<b>28</b>
What is the sanakey?.....	28
Indications .....	28
Fields of Application .....	28
Contraindications and Non-Applications .....	28
Side-effects .....	29
Safety Instructions .....	29
Briefing and Book of Medical Devices .....	30
<b>Scope of Delivery</b> .....	<b>30</b>
Optional Accessories .....	30
<b>Handling</b> .....	<b>31</b>
sanakey brief description .....	31
Description of the Functions of the Home Button.....	32
Description of the Touchscreen Controls.....	32
Inactive Display .....	33
Display Locking Times .....	33
Therapy Modes of sanakey.....	34
Overview of Main Settings (main menu) .....	34
Basic setting functions .....	35
Starting and stopping default therapy modes (therapy programs).....	35
Independent therapy mode selection .....	36
Characteristics of <b>Key-Phoresis</b> (Mode: KP Acid and KP Base).....	38
Device Settings.....	38
Charging the sanakey .....	39
Replacing the Changeable Electrode Head.....	40
<b>Care and Cleaning</b> .....	<b>42</b>
<b>Error Messages</b> .....	<b>43</b>
Recalibration.....	44
<b>Service and Repair</b> .....	<b>44</b>
<b>Loss of Guarantee and Warranty</b> .....	<b>44</b>
<b>Return and Disposal</b> .....	<b>45</b>
<b>Attachment: Technical Information</b> .....	<b>45</b>
Pulse Forms of Therapy Modes .....	45
Technical Data.....	46
Symbols and Pictograms.....	47
Results of performed EMC Tests.....	48

# Definition and General Information

## What is the sanakey?

sanakey is an adaptively regulated electrotherapy appliance.

The device uses the process of non-invasive electrical nerve stimulation which has been established as a successful therapy for many years, through the electrode's contact with the skin. This process is generally regarded as displaying extremely few side-effects.

sanakey can create bidirectional (in "Bioregulation") and galvanic (in "key-phoresis") forms serving as a base for the following indications and fields of application:

## Indications

The therapeutic aim of the sanakey is to reduce acute and chronic pain, improve blood circulation and perform passive muscle stimulation for the following fields of application:

## Fields of Application

- Nervous system
- Musculoskeletal system
- Digestive tract
- Urogenital area

The recommended treatment duration for the devices ranges from a few minutes to a maximum of 1 hour. The treatment is usually performed by moving the device on the skin.

sanakey is a medical product. It can be used by medical professionals as well as at home by laypersons who have been trained in its use by a medical professional.

## Contraindications and Non-Applications

In the following cases, sanakey may not be used or only in consultation with a doctor:

- Use prohibited on injured skin (e.g. open wounds and wounds which have not yet completely healed)
- Use prohibited in body orifices
- Use prohibited on carriers of active controlled implants (e.g. cardiac pacemakers)
- Use prohibited in the area of the uterus of pregnant women
- Use prohibited above tumours
- Use prohibited on patients with cardiac arrhythmia
- Use without consultation with a doctor prohibited on patients with malign diseases
- Use without consultation with a doctor prohibited on patients with uncontrolled epilepsy

## Side-effects

The **sanakey** shows extremely few side-effects. The following can, however, be regarded as side-effects:

Symptom intensification: Symptoms may intensify during the treatment period since the body is “dynamised”. In individual cases, this may be accompanied by stronger pain, muscle ache and other reactions, but they generally cease quickly. Should they last longer, you should interrupt the treatment for a few days and, if necessary, consult a doctor.

Like all metals, the steel of the electrodes may contain traces of nickel. Patients with pronounced nickel allergy may therefore experience skin irritation. In this case, medical professionals must decide whether to continue therapy based on a risk-benefit assessment.

## Safety Instructions

- Read the manual (this document) before you use the device for the first time.
- **sanakey** must be kept out of reach of children and stored accordingly. Numerous dangers might arise if the device gets into children’s hands.
- Use the **sanakey** in continuous operation only at ambient temperatures of up to 30° C max.
- Do not expose the **sanakey** to temperatures above 40° C and protect it against direct sunlight.
- Do not submerge the **sanakey** in water or hold it under flowing water.
- Do not dispose of the device in household waste, but according to special waste regulations.
- Do not open the device.
- Do not use it near inflammable substances.
- Only use the **sanakey** for the intended purpose.
- Only charge the device with the delivered charger.
- With regard to electromagnetic compatibility (EMC), the device also fulfills the stricter requirements regarding “home environment”, which also apply to many doctor and therapist practices. Nevertheless, the use next to other devices, also in a stacked form, should be avoided.
- In order to avoid EMC-related interference, only the power supply provided by the manufacturer and other original accessories are to be used.
- Portable RF communication devices (radio devices) of all types, including their individual components and all accessories, must not be operated within 30 cm of the device.
- Nevertheless, in very rare cases of high EMC interference, the system might switch itself off. In such a case, restart the device to continue therapy.
- Check the function of the device and components upon delivery:
  - function of the display
  - impulse transmission
  - charging function

# Briefing and Book of Medical Devices

You can receive a briefing into the device use

- through your medical product consultant (free of charge)
- through telephone support of a speaker of **KeyAkademie** (German only; free of charge)
- by visiting seminars of **KeyAkademie** (German only; professionals only; details: [www.keyakademie.de](http://www.keyakademie.de); fees apply)
- by in-house seminars of **KeyAkademie** (German only; fees apply)

Information for professionals: Safety-related checks have to be performed by professional staff according to German law (if the device is operated in Germany). If requested, the manufacturer can support through providing a protocol and fee-based service.

## Scope of Delivery

The scope of delivery of the sanakey includes:

- Hand-held device
- Standard changeable electrode head
- Charging device with mini USB connector (HNP06-USB)
- Instructions for use

## Optional Accessories

Highpro. Ideal for cross-dermatome treatment and therapy of meridians.

Trigger. Treatment of small-scale structures and muscle strands, tendons, trigger points.

Matrix. Specialised for fascia therapy and treatment of connective tissue.

Comb. For treatment of hairy areas.

Connector cable. Connection of various electrodes for long-term therapy.


Faszien-Flow. For treatment of fascia and trigger points, face and **key**-phoresis.

Tabletop base station. Secure storage with charging function.

## Handling

### sanakey brief description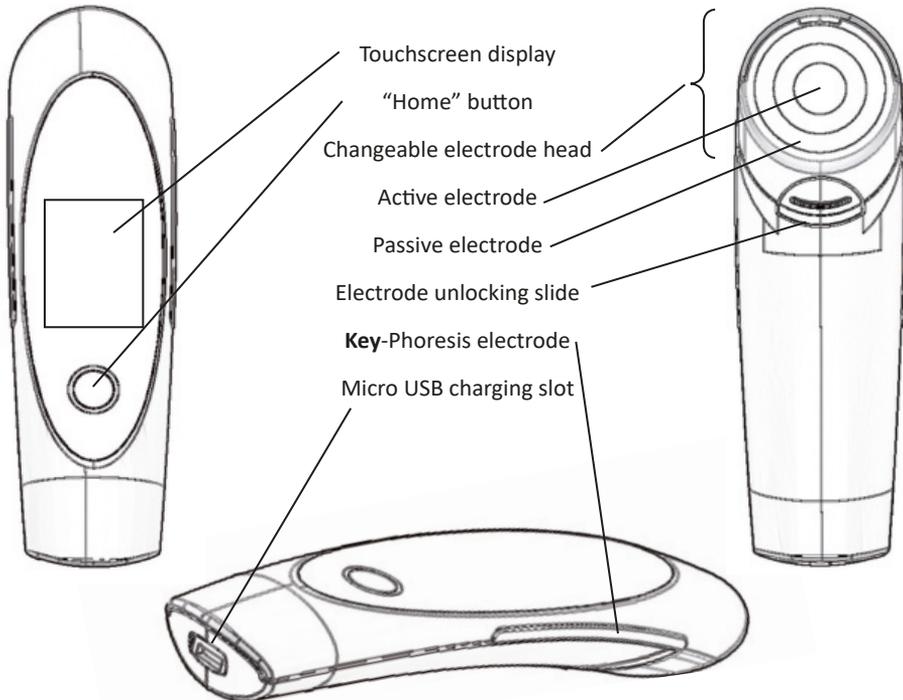

## Description of the Functions of the Home Button


The “Home” button is the only “physical” pushbutton on the **sanakey** and offers 4 functions:

1. **On/Off:** With the “Home” button, the device is switched on. To do so, press the button for two seconds. By pressing the button again for two seconds, the device is switched off again. Please wait at least 15 seconds before switching on the device again.
2. **Back to Main Menu:** The second function of the “Home” button is to return to the main menu by briefly pressing the button while the touchscreen is active.
3. **Reactivating of the Display:** If the touchscreen controls are not active, they can be reactivated by briefly pressing the “Home” button.
4. **Reset/Re-Calibration,** to be executed if a bluescreen is shown. See chapter troubleshooting.

## Description of the Touchscreen Controls

The display of the **sanakey** features touchscreen controls. If the energy setting or the therapy mode is to be changed, simply press the respective symbol or button on the display. In this description, the phrase “press” is often used regarding symbols/buttons. However, light touching is often sufficient. In general, touching the button directly triggers the intended function and is confirmed by an acoustic signal. In a list, however, functions are selected first with the arrow symbols and then started by another selection symbol (see description: Device Settings).

PLEASE NOTE: Handling the touchscreen is a matter of practice. After a few applications, the functions of the touchscreen will be familiar.

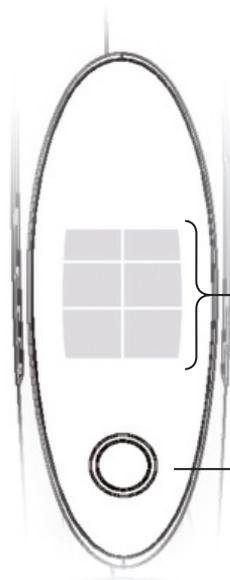The **sanakey** is controlled via the specially shaped control ellipse. This control ellipse is designed in such a way that accidental changes of the device settings can almost be excluded. The following controls are available:

Touchscreen controls and display  
(see section: Touchscreen controls)

“Home” button with on/off function

## Inactive Display

To prevent accidental activation of the touchscreen, the touchscreen functions are subsequently locked if they are not used for a specific period of time. If one section of the touchscreen controls is inactive, the following symbol is displayed at the bottom of the screen:

Additionally, the locked buttons turn white.

Briefly press the “Home” button once to reactivate the touchscreen display. If the display is activated, press the “Home” button once to return to the main menu.

## Display Locking Times


Already after 4 seconds, only the plus and minus buttons remain active to adjust the energy setting. If the energy setting is not changed for 9 seconds, all touchscreen controls are locked and the display brightness is reduced. Press the “Home” button to fully reactivate the touchscreen controls.

# Therapy Modes of sanakey

Use the sanakey only for its intended purpose. Note the contraindications.


Make sure that the sanakey is fully charged (see section: “Charging the sanakey”).

Press the “Home” button for approx. 2 seconds to switch on the sanakey. The following display indicates that the device is started up: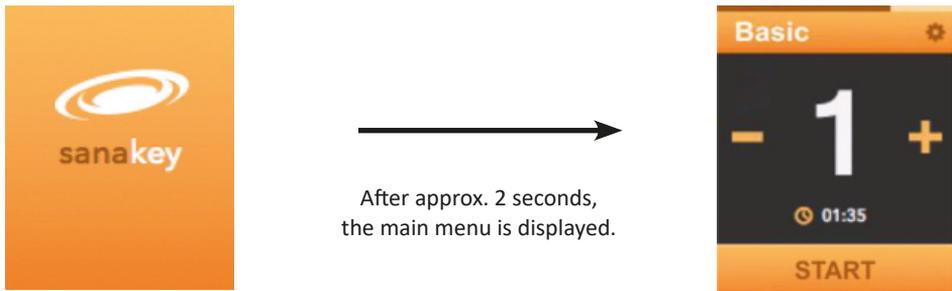

English

## Overview of Main Settings (main menu)

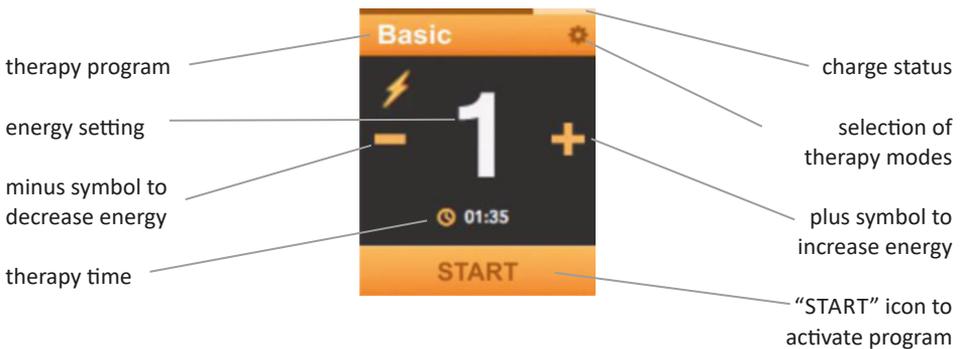If the lightning symbol is displayed below a therapy program, the output energy is applied at the electrodes.


## Basic setting functions

After the **sanakey** is switched on, the basic settings of the device are always active. In these settings, the “asymmetries” are searched as described in the therapy instructions. First, increase the energy setting by pressing the plus button. Press the minus button to decrease the energy setting. Pressing buttons is confirmed by an acoustic signal. The energy setting is indicated by the number in the centre. The energy setting can be set between 1 and 99. In basic settings, the energy always starts at 1. For the respective setting, please refer to the therapy instructions or contact your physician or therapist.


## Starting and stopping default therapy modes (therapy programs)

The **sanakey** can save default therapy modes. If a particular treatment was selected by a physician or therapist, this can be accessed again by pressing “START”. Press the “Home” button to return to basic settings.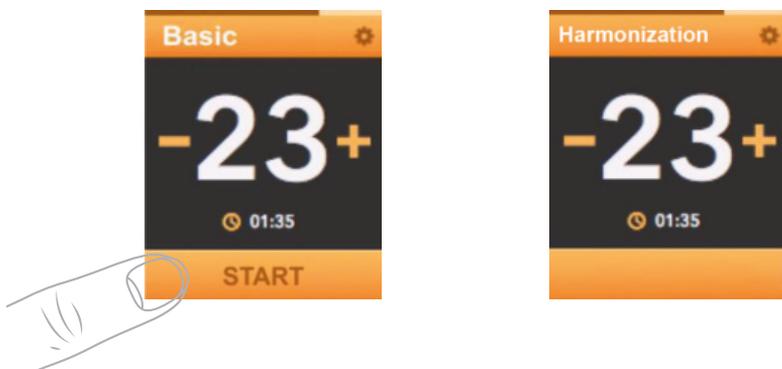


## Independent therapy mode selection

The different therapy modes may also be selected independently. We recommend selecting the therapy mode after consultation with a physician or therapist or after thorough consultation of the therapy instructions to ensure the best treatment success. To select the therapy mode, press the gear symbol. This symbol opens the selection menu.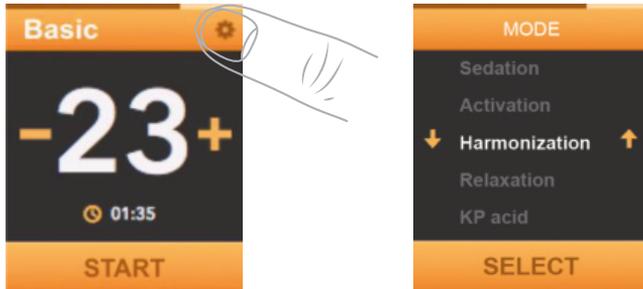

In this selection menu, the various modes (therapy programs) can be selected. To browse through the modes list, press the respective arrow buttons several times. The selected mode (therapy program) is highlighted.

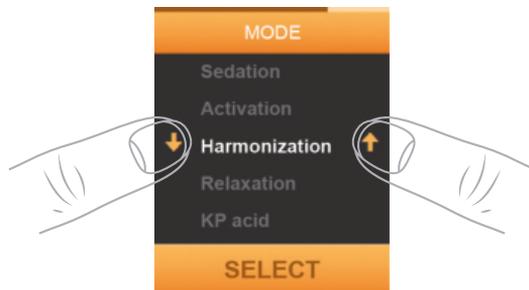To save this mode as default, press “SELECT” after selecting it. After pressing “SELECT”, the display automatically returns to treatment mode. The selected mode is now displayed on top. Even if the device is switched off, this default setting is saved and can be accessed by pressing “START” in the basic settings. Please note that the default energy setting is always 1.

CAUTION: Different therapy modes have a different intensity. The sanakey will apply slight automatic adjustment, however, the energy setting may still need to be adjusted manually.

## Characteristics of **Key**-Phoresis (Mode: KP Acid and KP Base)

KP is short for **key**-phoresis.

**Key**-phoresis is a special form of application using the characteristics of direct current. This enables the application of the galvanic treatment, which uses a low direct current.

The **sanakey** electrode is applied on the skin and the hand-held electrode (on the side) is required as counter electrode. It is naturally touched when using the device and the contact ensures that the electrode effect is applied. To close the circuit, contact with the ideally slightly moistened skin (also the hand) is required.

In phoresis mode, **sanakey** is used as an aid to bring in externals. The manufacturer's information in the leaflet of the external is decisive here. Please note their instructions for use as well as indications and contraindications of the external.

Choose the mode "KP Acid" if the external has an acid PH value and "KP Base" if the external has a basic PH value.

**Key**-phoresis is generally not felt. Nevertheless, the desired effect is achieved.

To carry out **key**-phoresis, increase the energy setting by pressing the plus button. Press the minus button to decrease the energy setting. The energy setting is indicated by the number in the centre. The energy setting can be set between 1 and 99. An energy setting that is barely noticeable or not noticeable should be selected. After selecting the energy setting, press "START", to stop the application, press "STOP".

## Device Settings

After switching on the device, the settings menu can be accessed by pressing and holding the gear symbol in the top right. The button can be released as soon as the settings menu is displayed. Press the arrow buttons at the left and right to select the respective settings from the list by pressing "SELECT". The following settings can be made:

- "**Language**": The options English and German are available. The respective option is selected from a list. Press "SELECT" to apply the set language.
- "**Sound**": The sound can be switched on or off. Warning signals like in case of low battery cannot be deactivated. The respective option is selected by means of two buttons for "Sound on" and "Sound off". The respective option is selected from a list. Press "SELECT" to apply the setting.


## Charging the sanakey

The **sanakey** indicates the current battery power during operation with a symbol (here illustrated on the display after starting the default settings):

Battery power: \_\_\_\_\_

Bright = Consumed power

Dark = Residual power

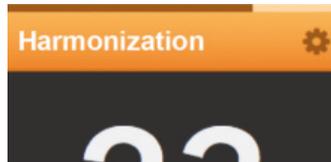Before the battery power is too low for ongoing operation of the **sanakey**, an acoustic signal can be heard. If the battery power is low, the **sanakey** cannot be started. Remove the USB plug and connect the **sanakey** to the charging cable.

The scope of delivery includes a battery charger with micro USB connector. To charge the **sanakey**, plug the charger in a suitable power socket and connect it with the **sanakey**. The device has to be switched off. It cannot be used during charging. During charging, the following screen is displayed (after a short time, the display is switched off during charging to save power and to accelerate the charging process. For information on the charging process, press the "Home" button to wake up the display.):

**sanakey** is charging

**sanakey** is fully charged

## Battery life

The battery has been specially adapted to the characteristics of the sanakey. It is **not removable**. Please note the section: "Return and Disposal". To ensure maximum battery life, please note:


- Should the batteries discharge entirely, the sanakey is broken. Charge the battery if it is running low.
- Switch off the sanakey if it is not used.
- Charge your device with approved charging accessories only.
- Do not switch the sanakey on and off in quick succession.

The minimum operating time of the battery should be at least 1 hour depending on the applied therapy modes and the energy setting. If the minimum operating time is clearly below this value, the technical service may be contacted to have the battery replaced (subject to a charge).


## Replacing the Changeable Electrode Head

The electrode on the bottom of the sanakey is attached to a changeable electrode head. This changeable electrode head can be replaced by specific optional changeable electrode heads considerably expanding the therapy spectrum.


To replace the changeable electrode head, slide back the unlocking slide.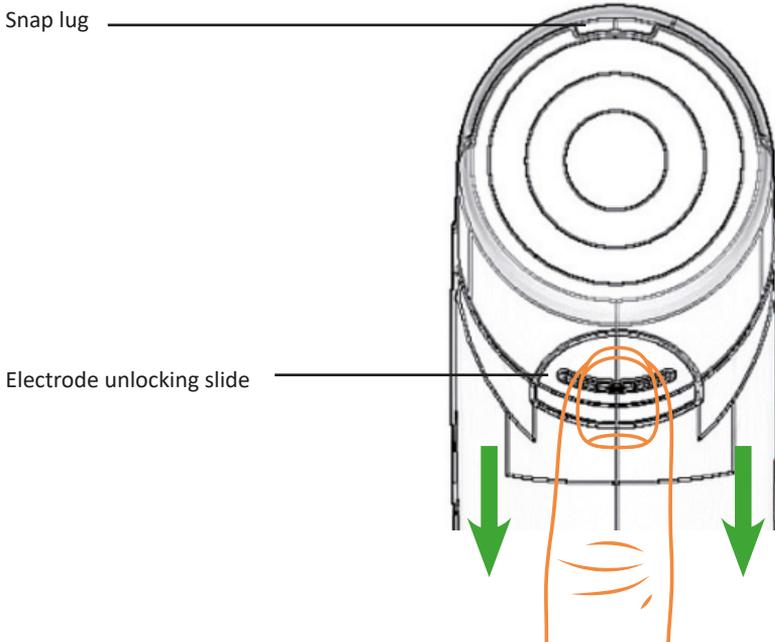

Afterwards, take the changeable electrode head out of its guide by lifting it at the side of the unlocking slide.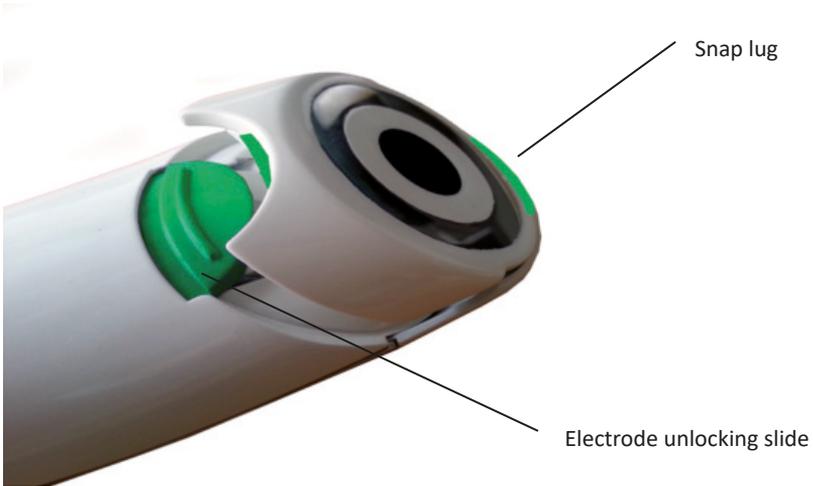

To insert a different changeable electrode head, first slide the recess in the changeable electrode head under the snap lug while tilted. Lower the rear end and engage it with the unlocking slide.

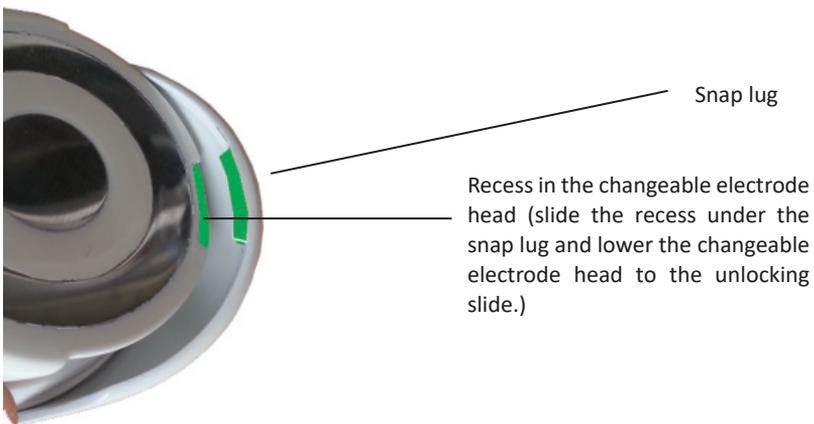## Care and Cleaning

The sanakey design is robust and durable. Nevertheless, we **strongly recommend** treating the sanakey with great care like any other technical device. Insufficient or incorrect care and cleaning may void the warranty. The same applies to gross negligence.

It is recommended to clean and disinfect **both the device and the changable electrode head part** after each therapy treatment and before the beginning of a new therapy treatment with a different patient.

### Please proceed as follows:

- Switch off the sanakey after each use.
- Recommendation: Clean the sanakey after each use.
- Apply cleaning agent on a moist soft piece of cloth.
- Wipe off the device, also in the area of the snap lug.
- Remove and wipe off the changeable electrode head.
- Wipe afterwards with a moist cleaning cloth without cleaning agent.

Isopropyl alcohol is to be used for disinfection. Alternative procedures must align with national requirements and the Robert-Koch-Institute's recommendations (Germany). **Avoid the use of ethanol, especially on the parts (display) made from acryl!**

Please note that the sanakey is not waterproof. For this reason, all parts should only be handled with a piece of cloth moistened with cleaning agent. If required, the device can be cleaned extensively with the following means:

- Soap solution (dilution)
- Grease detergent (dilution)

Ensure that no humidity penetrates the interior of the device.

# Error Messages

The **sanakey** offers various self-test and monitoring mechanisms. These monitor the integrated hardware and the function of the firmware.

Error	Likely Cause and Solution
device shows a blue screen	<p>If an error is recognized by the monitoring mechanism, the application is interrupted immediately and a blue screen with an error message is displayed.</p> <p>The device is automatically switched off after 3 seconds with an acoustic interval signal. Press the “Home” button to display the error message.</p> <p>Recalibrate the device (see page 46 for a detailed instruction on how to recalibrate).</p> <p>The device can be switched on again with the “Home” button. The self-test after switching on the device must not recognize any errors to continue working with the <b>sanakey</b>.</p> <p>If this is not the case, the device is switched off again. In this case, the device should be sent to the technical service.</p>
Device cannot be turned on <i>or</i> Device turns itself off automatically	<p>If the device cannot be switched on, the screen remains black or can only be seen briefly, please check the battery. To do so, connect the included charger.</p> <p>If the loading screen is displayed, the battery is charged (refer to page 41).</p>
Temperature Error	<p>The use temperature of <b>sanakey</b> is exceeded or fallen below minimum.</p> <p>Wait until the device has cooled down or warmed up. It can be turned on again afterwards.</p>
Damage	<p>In case of visible damage at the device, immediately stop using the <b>sanakey</b>.</p> <p>It is recommended to return the device to the manufacturer to have it checked.</p>

## Recalibration

PLEASE NOTE: Recalibrate the device only with a fully charged battery!

A blue screen with an error message “go to service or reset device” is displayed. Restart the device. Press the “Home” button again and hold it for 10 seconds until a blue screen is displayed with an acoustic signal. Release the “Home” button and wait for the device to shut down automatically.

**CAUTION!!!** Before switching on the device again, make sure that there is no contact between the electrodes and the patient, you or any other conductive objects.

Switch on the device and wait for the device to calibrate automatically. This may take up to one minute.

**CAUTION!!!** In this time, there must be no contact between the electrodes and the patient, you or any other conductive objects.

After calibration, the settings menu is automatically displayed. Here, settings like the language may be changed as usual. After pressing the "Home" button, the device may be used as usual.

## Service and Repair

The manufacturer is only considered to be responsible for safe operation of the **sanakey** if repairs, updates and other modifications are carried out by the manufacturer himself and the device is applied in accordance with its intended use.

The battery may only be changed by the manufacturer.

## Loss of Guarantee and Warranty

The guarantee/warranty does not cover consumables and wear parts. The following attempts at the **sanakey** lead to loss of guarantee and warranty:

- Opening
- Manipulation of the hardware or firmware
- Combination with other technical means, e.g. foreign electrodes.
- Charging with a foreign, non-original charging cable
- Unintended use of the device
- Gross negligence (water ingress into the housing, fall, etc.).

## Return and Disposal

Batteries and rechargeable batteries may not be disposed of with household waste. You are legally obliged to return used and rechargeable batteries.

Since the **sanakey** is equipped with a rechargeable battery, please dispose of the entire device at collection points that are connected to the communal return system. Do not open the device. The manufacturer is also obliged to take back the device.

There are free collection points for used electrical devices in your area for their return and different collection points for reusing broken devices. You can get the addresses from your city or local government. If your **sanakey** contains personal data, you are responsible for deleting it yourself before returning it. You can find more information at [www.elektrogesetz.de](http://www.elektrogesetz.de).

The above information is valid for Germany. Please dispose of the **sanakey** according to national law in your country.

## Attachment: Technical Information

### Pulse Forms of Therapy Modes

Basic:	Standard frequency - all other parameters inactive
Calming:	High frequency in combination with amplitude modulation
Activation:	Low frequency - all other parameters inactive
Harmonisation:	Deviating standard frequency with high time interval and intensity
Relaxation:	Deviating standard frequency with different time interval and intensity
KP Acid:	<b>Key</b> -phoresis with galvanic treatment for generation of an ion flow from - to + to bring in acid externals
KP Base:	<b>Key</b> -phoresis with galvanic treatment for generation of an ion flow from + to - to bring in basic externals

# Technical Data

## Contact details

Manufacturer: **Keytec GmbH**  
Steppacher Str. 32a  
86420 Diedorf  
Germany

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## sanakey hand-held device

Dimensions: 185 x 51 x 28 mm (L x W x H)  
Weight: 140 g  
Ambient temperature: -25 ... 50° C transport and storage  
0 ... 30° C operation  
Ambient humidity: max. 93 % rel. humidity, non-condensing  
Ambient air pressure: 700 hPa ... 1060 hPa  
Protection class: IP 42  
Charging voltage: 5 V +/- 0.5 V  
Max. charging current: 500 mA  
Max. charging time: 180 min  
Battery capacity: 800 mAh  
Typ. operating time: 6 h (depending on the treatment mode and energy setting)  
Min. charging cycles: 500

Output value: a) Pulse operation: < 10 mA (rms) at 10 kOhm  
(depending on the operating mode)  
Pulse energy max. < 1 mJ/pulse at 10 kOhm  
Energy setting adjustable in 100 settings

b) **Key**-phoresis DC operation: max. 1.2 mA at 10 kOhm  
U<sub>max</sub> <= 12.5 V  
Power setting adjustable in 100 settings  
0 ... 1 mA 50 settings [20 µA/setting]  
1 ... 10 mA 50 settings [180 µA/setting]

The device is not disturbed by wireless communication equipment (e.g. mobile phones) nor causes disturbances.


### Charger (original accessory) (Type: HNP06-USB-L6)

Input voltage range:	90 ... 264 VAC
Input frequency:	47 ... 63 Hz
Input current:	max. 300 mA (rms) at 90 VAC, 50 mA (rms) at 230 VAC
No-load loss:	0.17 watts
Output voltage:	5V +/- 0.5 V
Power output:	6 watts

## Symbols and Pictograms

The sanakey is labelled with the following symbols and pictograms:

Name plate:

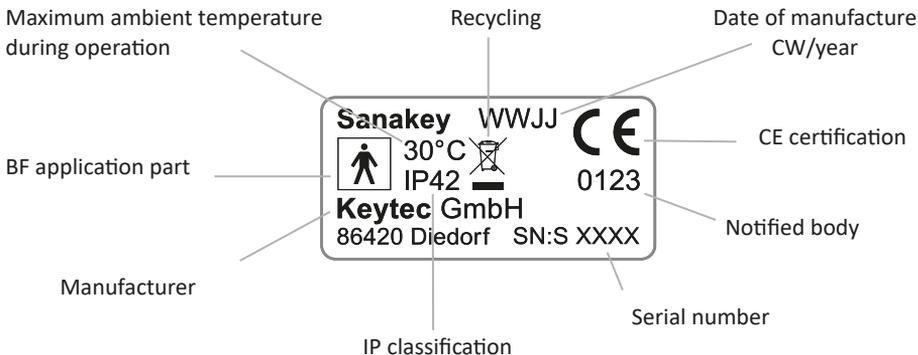Safety information:

**Caution!** Instructions for use and all accompanying documents must be observed!

# Results of performed EMC Tests

## Emission Tests in accordance to:

Norm: CISPR 11

Class: B

Group: 1

Electromagnetic Radiation Disturbance      30 MHz – 1 GHz      passed

## Immunity Tests and their Test- Levels

Electrostatic Discharge (ESD)      Contact Discharge: + 8 kV      passed

Air Discharge: +15 kV      passed

Electromagnetic Field      10 V/m      passed

Power frequency magnetic field      30 A/m      passed

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